


.....

ACRYLIC GLOVE DISPENSER

.....

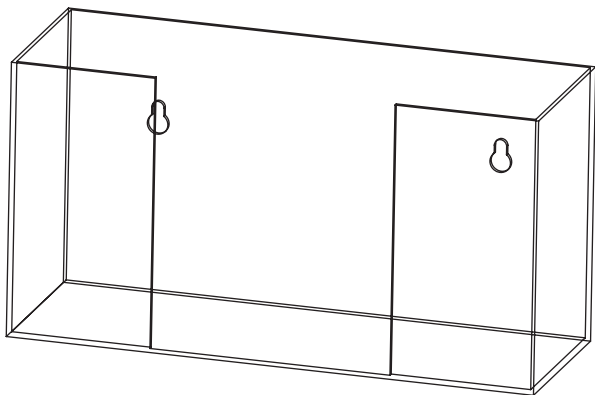

USER GUIDE

SKU: ADI902-SERIES



Thank you for your purchase.



Viva Essentials maintains a strong practice of ongoing development of products and commercial research.




We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.


Table of Contents

- 2 Acrylic Glove Dispensers
- 3 Parts List
- 3 Hardware List
- 3 Tools Required (Not Included)
- 4 Assembly Instructions
- 7 Warranty
- 8 Notes


Acrylic Glove Dispenser

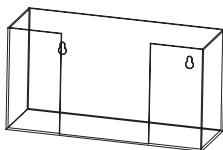


ADI902-01
Single
Box Capacity

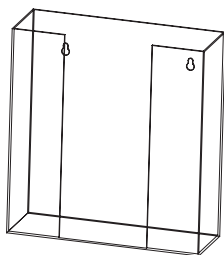

ADI902-02
Double
Box Capacity

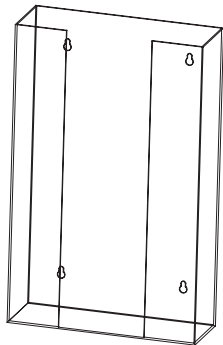

ADI902-03
Triple
Box Capacity

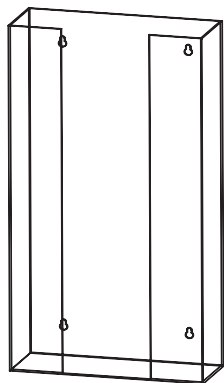


ADI902-04
Quad
Box Capacity

Parts List


A

Acrylic
Glove
Dispenser

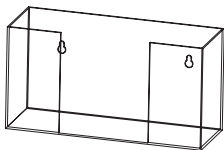

Hardware List

1

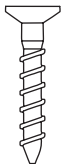

Flat Head
Screws

2




Plastic
Anchor

3




Double Sided
Tape (x2)


Tools Required (Not Included)

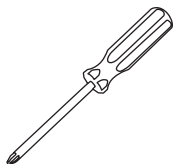

Phillips Head
Screwdriver

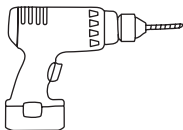

Drill and
Drill Bit

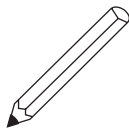


Pencil

Assembly Instructions


Step 1:

- A.** Pick a vertical, clean, and smooth mounting surface.
- B.** Ensure there is sufficient clearance above the Acrylic Glove Dispenser (A) to allow for easy refilling of gloves.

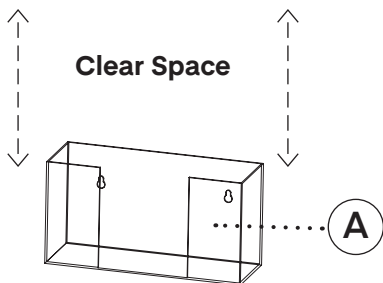


Step 2:

- A.** Use the back of the Acrylic Glove Dispenser (A) as a template.
- B.** With a pencil, mark the locations of the holes to be drilled.
- C.** Drill two holes for the single or double box capacity dispenser, or four holes for the triple or quad box capacity dispenser. Ensure the holes are not drilled too large.

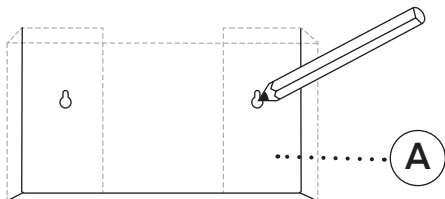


Step 3:

- A.** Insert Plastic Anchors (2) into the drilled holes.
- B.** Place the Acrylic Glove Dispenser (A), over the drilled holes, making sure the holes on the dispenser line up.
- C.** Screw in one of the Flat Head Screw's (1) through your Acrylic Glove Dispenser (A) into the Plastic Anchors (2).
- D.** Repeat the above steps for the remaining two Washers (3) and Flat Head Screws (1).

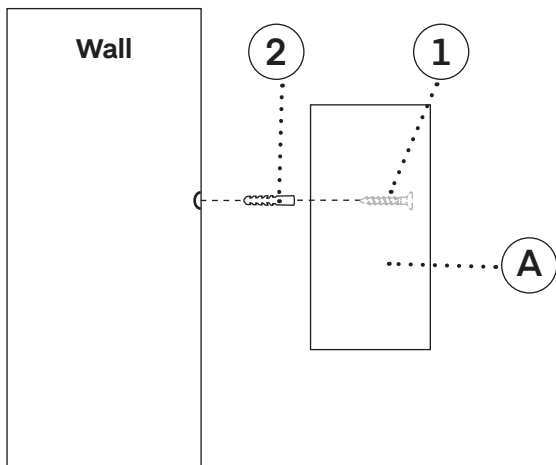


Step 4:

- A.** If you choose not to use the included screws and opt for the Double-Sided Tape (3):
- a.** Remove the protective paper from one side of the first tape strip and apply it firmly to the back of the dispenser. Hold in place for 30 seconds.
 - b.** Repeat the same process with the second tape strip.
- B.** Once both strips are applied:
- a.** Remove the protective paper from the exposed side of both strips.
 - b.** Position the dispenser on the desired surface and press firmly. Hold in place for 30 seconds to ensure a secure bond.

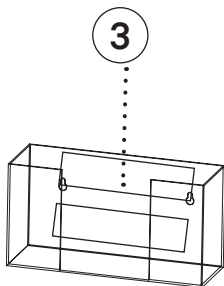

Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 1 year from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.

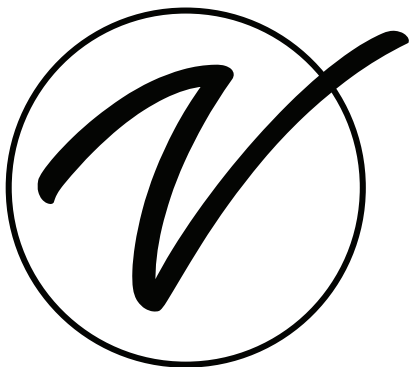

ADI902-SERIES

© 2025 Viva Essentials

vivacomfort.com