


# XRO SERIES POWER HI-LO EXAM TABLE

OPEN BASE, FLAT TOP CHANGING TABLE

.....

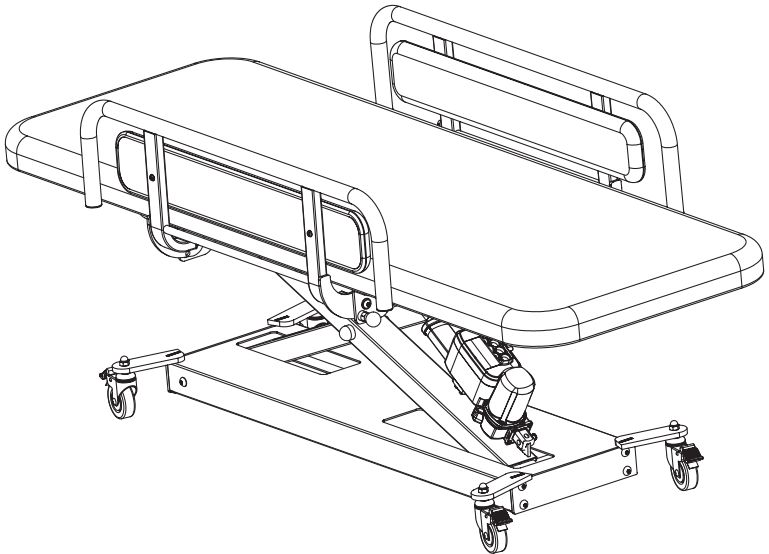

**USER GUIDE**

**SKU: ADI903-19-SERIES**



# Thank you for your purchase.



**Viva Comfort maintains a strong practice of ongoing development of products and commercial research.**



**We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.**

# Table of Contents

## **4 Specifications**

## **5 Safety Warning**

6 Risk of Collapse

6 Risk of Accidental Injury

6 Infection Risk

6 Risk of Damage to the Table Surface

7 Hardware Accessories Rusting Risk

7 Other Precautions

## **8 Preparation and Inspection Before Use**

## **9 How To Use**

9 Power Connection

9 Height Control

9 Table Movement

10 Installing the Guardrails

11 Lowering and Raising the Guardrails

12 Replacing Fixed Table Legs

## **14 Care & Maintenance Guide**

14 Leather Maintenance

14 Maintenance of Metal and Parts

14 Regular Maintenance


## **15 Warranty**

# Specifications

ADI903-19-SERIES

## **Xro Series Power Hi-Lo Exam Table with 1-Section Top**

Open Base, Flat Top Changing Table

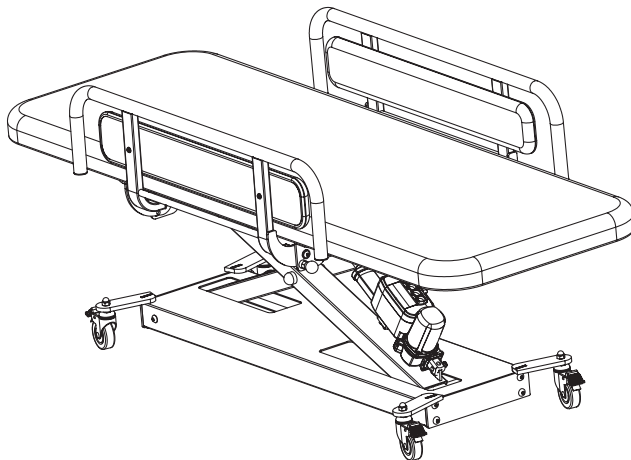

**Dimensions:** 17-33" H x 27" W x 71" L






**Net Weight:** 168 lbs.

**Lifting Capacity:** 440 lbs.


**Working Weight:** 550 lbs.

# Safety Warning


The user is responsible for preventing damage to this product and injury to patients and/or users.

	Warning: Indication of potential personal injury.
	Warning: Be careful of pinching fingers.
	Reminder: Keep away from moisture. This device needs to be moisture-free and waterproof.
	Reminder: Always place the packaging box in the correct direction during transportation, storage, and handling to avoid damage and safety problems caused by incorrect placement.
	Reminder: This product is fragile and needs to be handled with care.

## Risk of Collapse

 If the patient's weight exceeds the working load, the treatment table may collapse if the patient's weight is not evenly distributed on the table surface.

## Risk of Accidental Injury

 When the patient is lying in bed, raise the slide rails and make sure they are locked.

## Infection Risk

When cleaning, please wear gloves, masks, and eye protection to prevent the cleaning spray from rebounding from the contaminated surface.

## Risk of Damage to the Table Surface

- The ambient temperature requirement for using and storing the equipment is: 0-104°F. If the equipment is exposed to an environment exceeding the specified temperature for a long time, it may cause the risk of cracking and peeling of the leather surface.
- The humidity requirement for equipment transportation, use and storage environment is: 20%~80%. If the equipment is exposed to an overly dry or humid environment for a long time, the leather surface may crack or peel.
- The leather of the equipment table is prohibited from contacting acidic or alkaline substances. If a non-neutral detergent is used to scrub the table surface, the leather surface may crack or peel prematurely, reducing its service life.
- The leather of the table surface is prohibited from contacting liquid water. Liquid water may penetrate the leather, causing the sponge inside the leather to degenerate, affect its use, or cause the leather surface to crack or peel.

# Hardware Accessories Rusting Risk


- The humidity requirement for equipment transportation, use, and storage environment is 20%~80%. If the equipment is exposed to an overly humid environment for a long time, the hardware accessories may rust.
- The metal parts of the equipment are prohibited from contacting acidic substances; otherwise, they may cause damage to the paint surface and then rust.
- The metal parts of the equipment are prohibited from contacting liquid water, which may cause metal materials to rust.

## Other Precautions

- It is prohibited to place near a fire source.
- Do not place it in direct sunlight.
- Do not place it in high-temperature places such as near heaters.

# Preparation and Inspection Before Use


- Confirm whether a product manual accompanies the product.
- Confirm whether the components are deformed or damaged.
- If any abnormalities occur during installation and use, please get in touch with the dealer.
- Please place the table on a level surface.
- After positioning the table, before use, be sure to lock the casters to prevent them from rotating or the medical bed from moving.

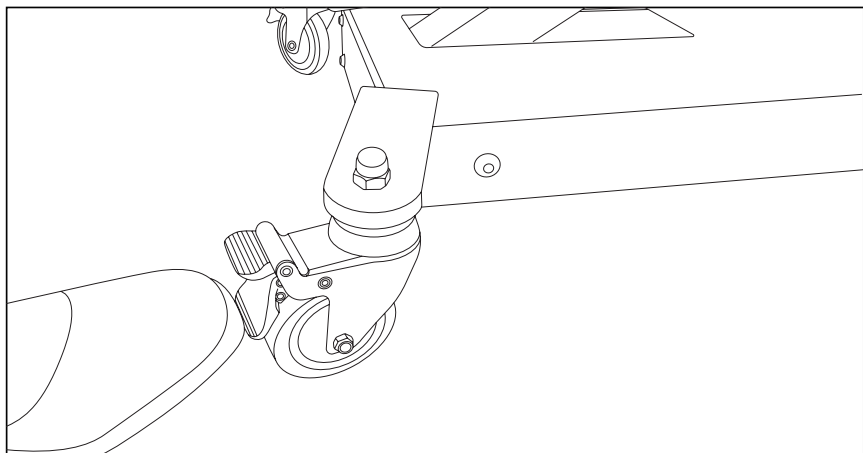

# How To Use

## Power Connection


- A.** Remove the table from the packaging and position it on a flat surface.
- B.** Connect the table's power cord plug to an outlet.

 **Reminder:** Please disconnect the power supply when the designated doctor or operator leaves.

## Height Control


### A. Foot Pedal Control

- a.** Plug the foot pedal into the motor.
- b.** Operate the foot control to raise or lower the table height, as shown.

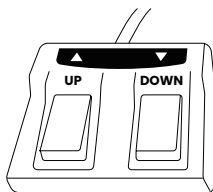


### B. Hand Control (Add-On)

- a.** Plug the foot pedal into the motor.
- b.** Operate the hand control to raise or lower the table height.

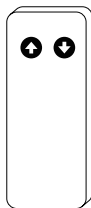


## Table Movement

 **Note:** When moving the medical bed on inclined surfaces, exercise extra caution to prevent the bed from colliding with walls, other individuals, or obstacles, as such impacts may damage the leather bed surface.


## Installing the Guardrails (Add-On)

**A.** Install the guardrail base under the table upholstery, and make the mounting holes of the guardrail base correspond to the reserved mounting holes under the bed.


**B.** Use a wrench to rotate the 1/4" x 20mm bolts clockwise. After all bolts are installed, tighten the bolts again.

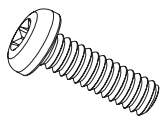

1/4" x 20mm  
Bolt (x16)

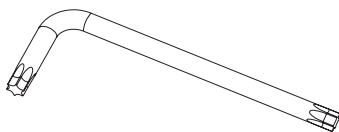


Wrench

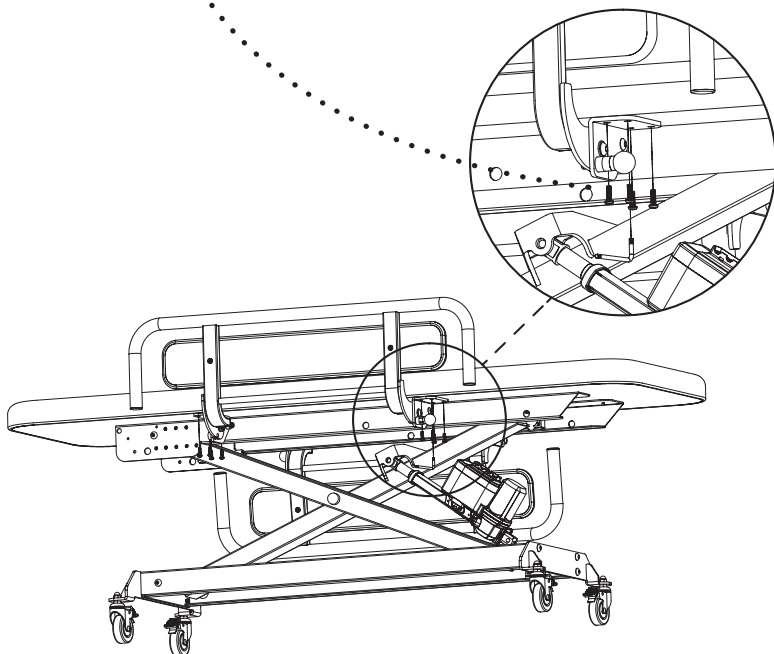


## Lowering the Guardrail

- A.** Pull out the round latch ball and flip the guardrail downward until it is vertically downward.
- B.** Insert the latch ball back to lock the guardrail.

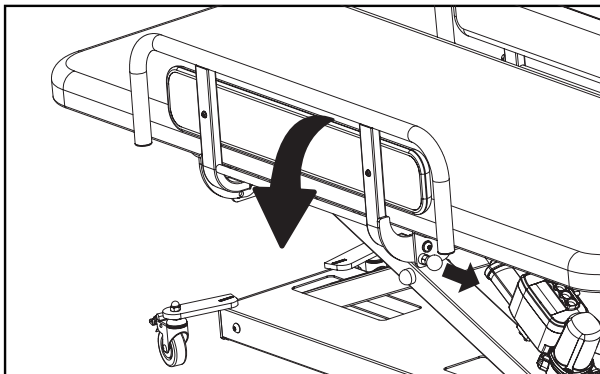


## Raising the Guardrail

- A.** Pull out the round latch ball and flip the guardrail upward until it is vertically upward.
- B.** Insert the latch ball back to lock the guardrail.

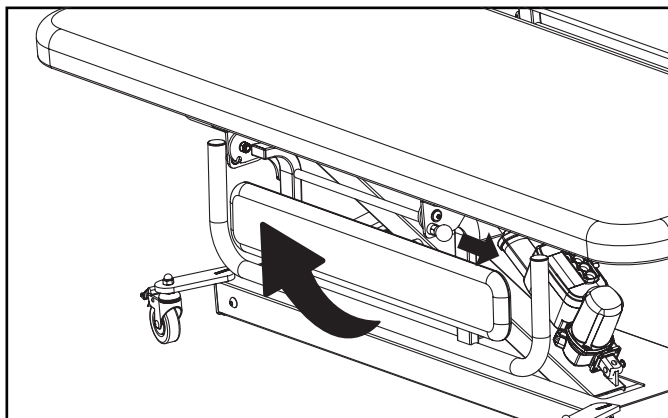


## Replacing Fixed Table Legs (Add-On)


**A.** Use a wrench to rotate the bolts on the steel plates on both sides of the table base counter clockwise and remove all four bolts.

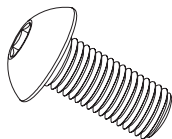

M8x16mm  
Bolt (x16)

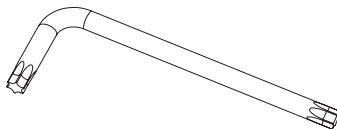


Wrench

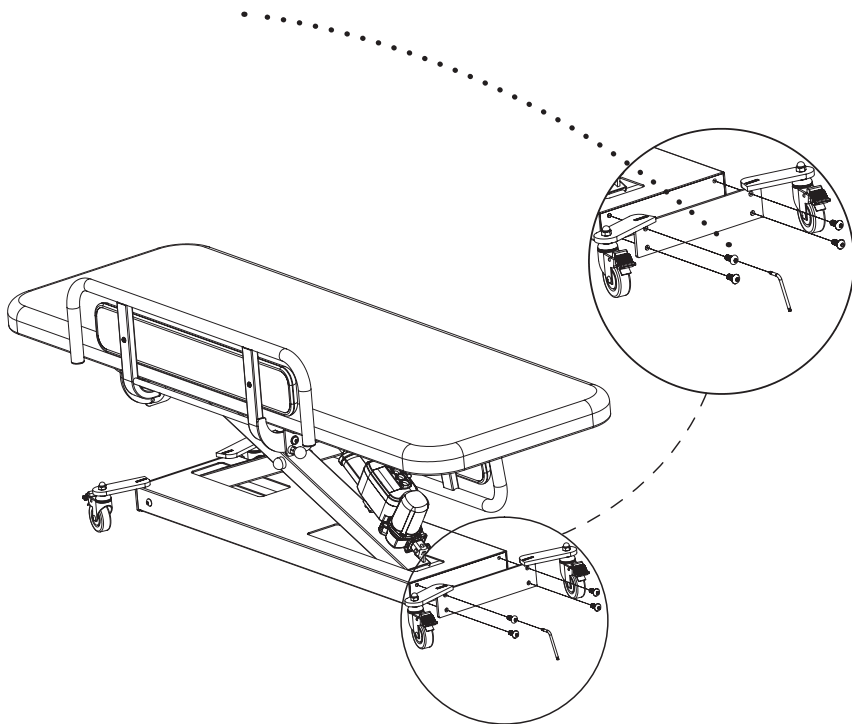

**B.** Align the reserved installation holes of the replacement fixed table legs with the reserved installation holes of the table base.

**C.** Use a wrench to rotate the M8x16mm bolts clockwise. After tightening all four bolts, tighten each bolt again to reinforce it.

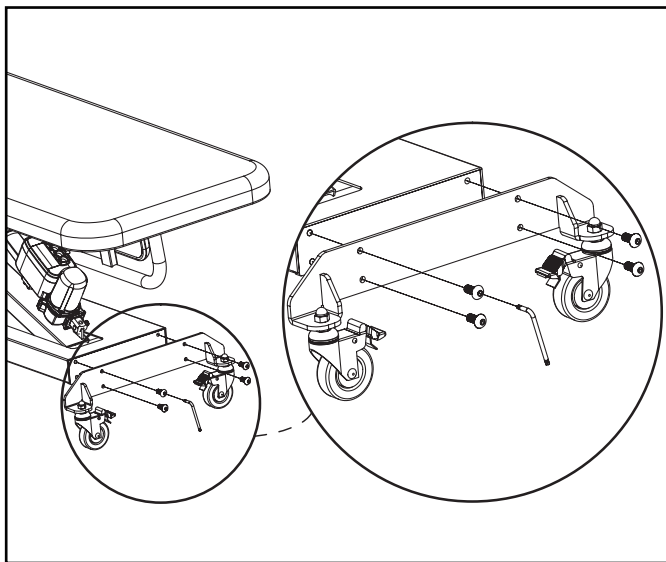

# Care & Maintenance Guide

## Leather Maintenance

- a.** It is recommended to use a small amount of neutral detergent/cleaner or wipe with a dry towel when wiping the leather, then wipe it clean with a dry cloth, dry it, and place it in a ventilated place to dry.
- b.** If disinfection is necessary, it is acceptable to wipe it with hypochlorous acid disinfectant or alcohol below 70% concentration. 84 disinfectant or other highly corrosive disinfectants are prohibited for disinfection; otherwise, they may accelerate the aging of the leather, causing the leather to change color, deform, harden, become brittle, or even peel off the surface.
- c.** When not used for a long time, it is recommended that the table be covered with a table cover and stored in a dry environment. Avoid direct sunlight, rain, humidity, and other environments.
- d.** Too high or too low-temperature environments may accelerate the aging of leather.
- e.** Sharp objects are prohibited from touching the leather surface.

## Maintenance of Metal and Parts


- a.** A dry towel is recommended to wipe plastic, metal frame and metal parts.
- b.** If disinfection is necessary, it is acceptable to use hypochlorous acid disinfectant or alcohol with a concentration below 70% to the maximum extent possible. Using BA-SI disinfectant or other highly corrosive disinfectants is prohibited for disinfection. Otherwise, the paint may be damaged.

## Regular Maintenance

- a.** Inspect the table frame every 3 months to check the stability of the table frame when it is pushed horizontally, vertically, or laterally. If there is any shaking, tighten the connecting bolts.

# Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 3 year from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.

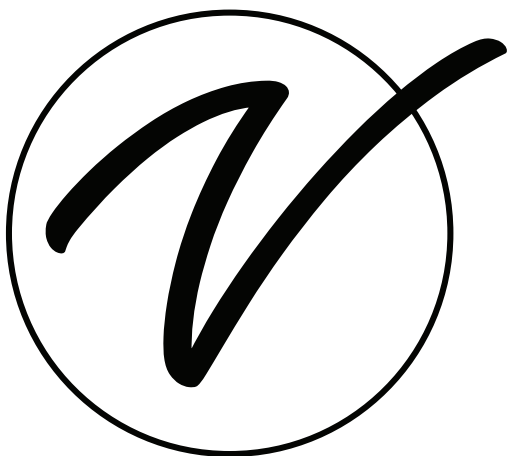

**ADI903-19-SERIES**

© 2025 Viva Comfort

**[vivacomfort.com](http://vivacomfort.com)**