




.....

WALL-MOUNTED FOLDING THERAPY TABLE

.....

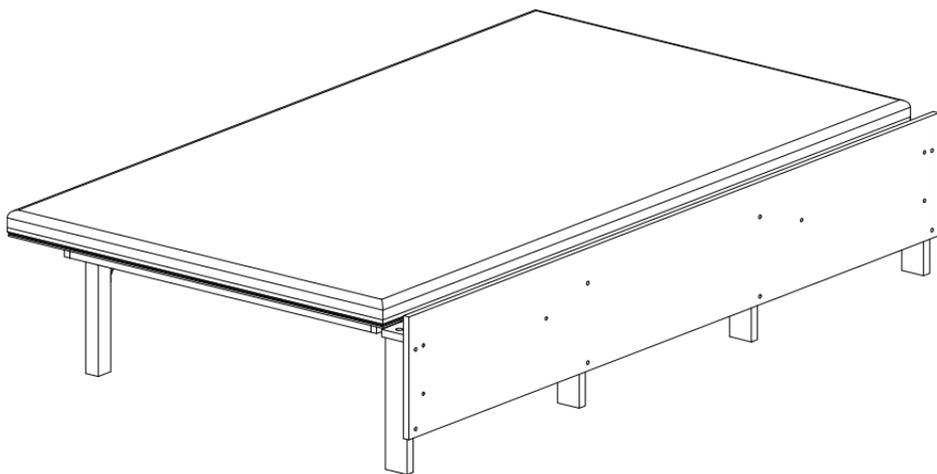

USER GUIDE

SKU: ADI996-06-SERIES



Thank you for your purchase.



Viva Comfort maintains a strong practice of ongoing development of products and commercial research.




We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.

Table of Contents

- 2 Parts List
- 3 Hardware List
- 4 Tools Needed (But Not Included)
- 5 Assembly Instructions
- 13 How to Fold and Mount
- 14 Warranty


Parts List

A

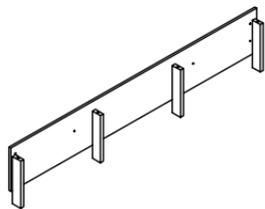


Mounting Board

B

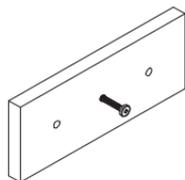

Chain Block


C

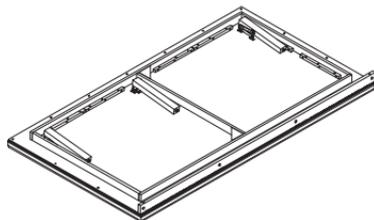

Mattress

Hardware List

1

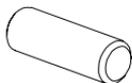

Wooden Dowel (8x30) (x2)

2



Bolt (M6x40mm) (x4)

3




Spring Washer 6.5 (x4)

4




Flat Washer 6.5 (x4)

5

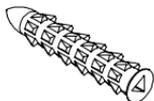


Plastic Anchor (10x40) (x8)

6

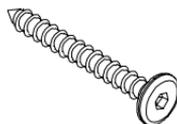

Screw (M6x50mm) (x8)

7

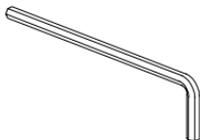


Allen Wrench

8




Chain


Tools Needed (But Not Included)

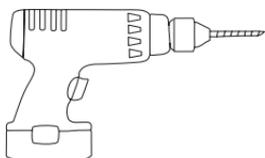


Drill

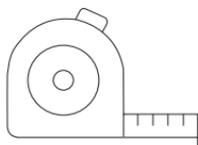


Tape Measure

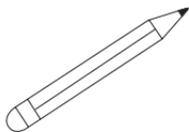


Pencil

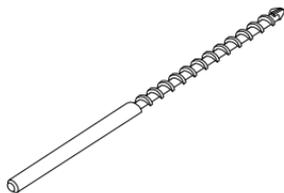

Drill Bit

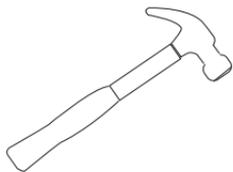

Hammer

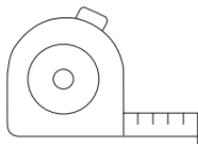

Tape Measure

Assembly Instructions


Note: It's recommended to utilize two people for the below steps to prevent strain or injury.

Step 1:

- A. Locate on your wall where you would like your therapy table.
- B. Using a stud finder, try to identify the primary beams within your wall to ensure a stronger and more secure installation.
- C. Using a level and pencil, mark the eight holes you will need to drill on the wall.


a. Below is a drilling guide that shows you the heights at which your drill marks should be placed.

b. If you're uncomfortable with this step, contact a contractor.

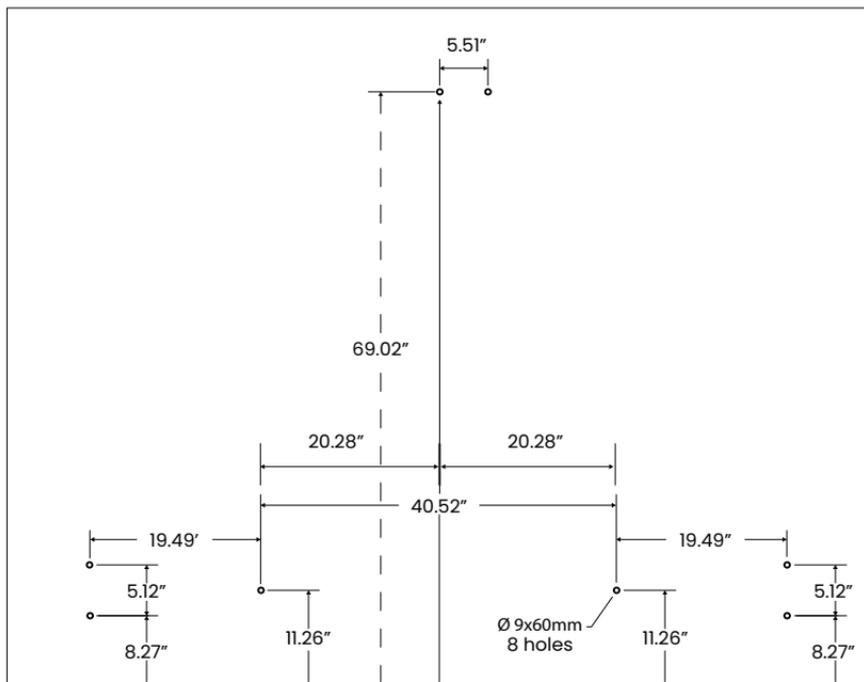


Step 1 (Continued):

- D.** Drill all the premarked areas to $\varnothing 9 \times 60$ mm and insert eight Plastic Anchors (5).
- a.** Gently tap the anchors in with a mallet or hammer.

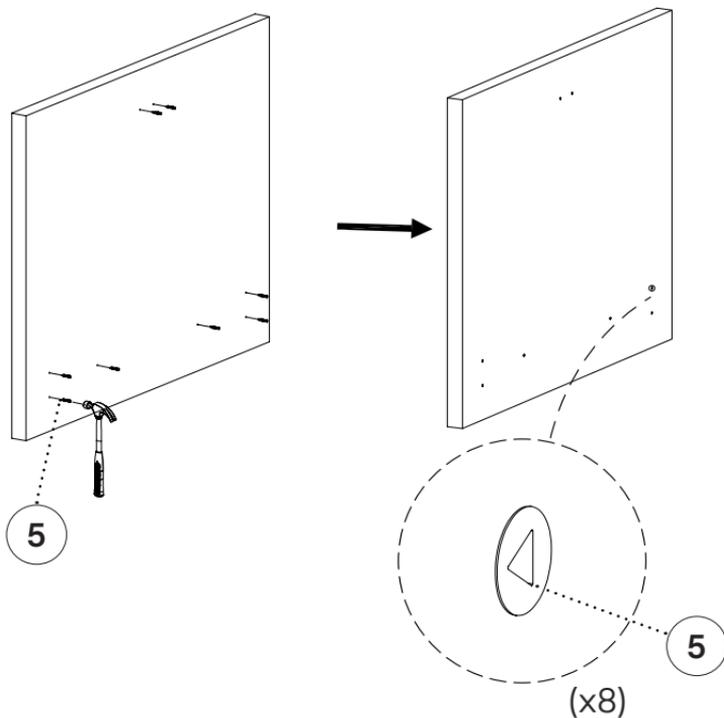


Step 2:

- A. Locate the Chain Block (B) and two Screw's (6).
- B. Line the Chain Block (B) holes up with the two highest predrilled holes from Step 1.
- C. Insert a Screw (6) into one of the predrilled holes and secure it in place with the included Allen Wrench (7).
 - a. Repeat the above step for the second Screw (6).

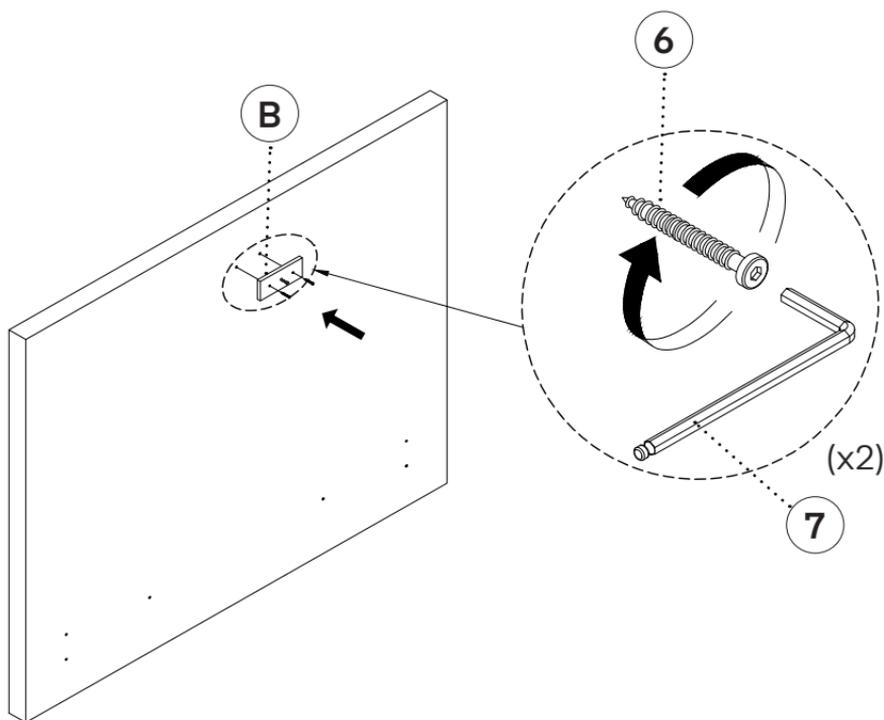


Step 3:

- A. Utilizing two people, locate the Mounting Board (A), and align it with the Plastic Anchors (5) you installed in Step 1, as pictured.
- B. Insert six Screw's (6) into the holes on the Mounting Board (A), as pictured.
- C. Secure them in place with the included Allen Wrench (7).

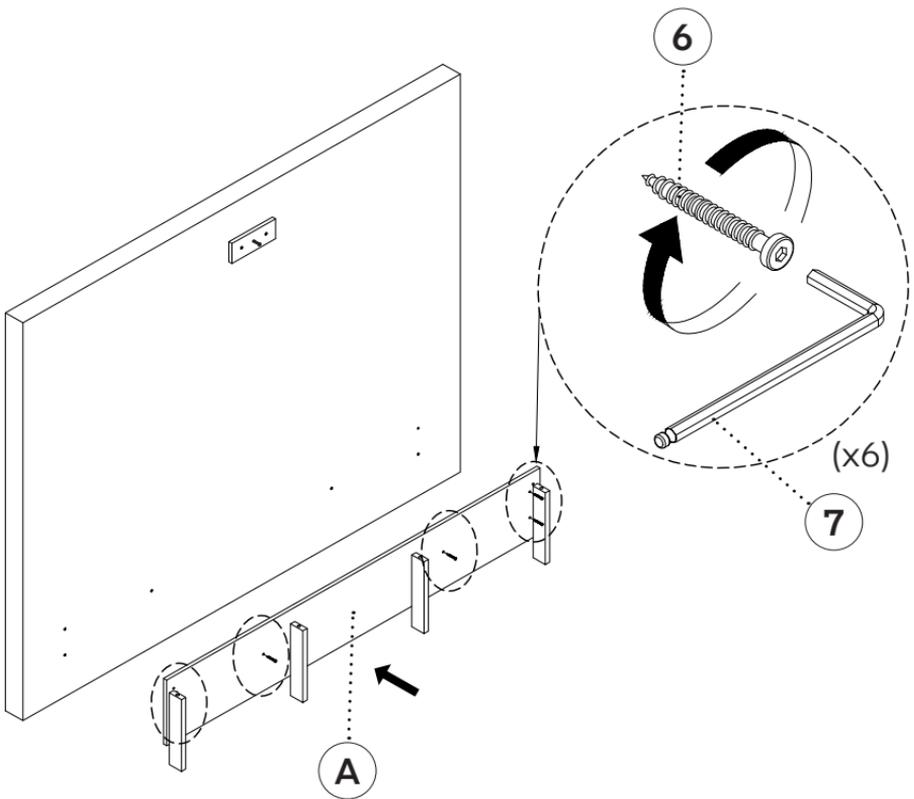


Step 4:

- A.** Utilizing two people, locate and lay the Mattress (C) onto a clean surface.
- B.** Fully extend the three legs, as pictured.

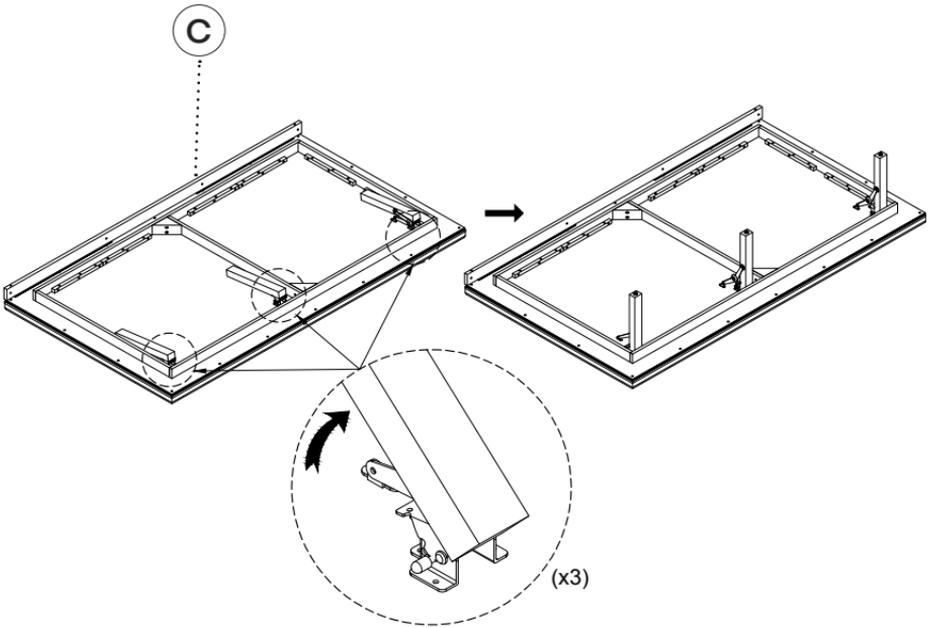


Step 5:

- A.** Insert a Wooden Dowel (1) into the hole on the leftmost side of the Mounting Board (A), as pictured.
- B.** Repeat the above step for the second Wooden Dowel (1), but insert it into the rightmost hole, as pictured.

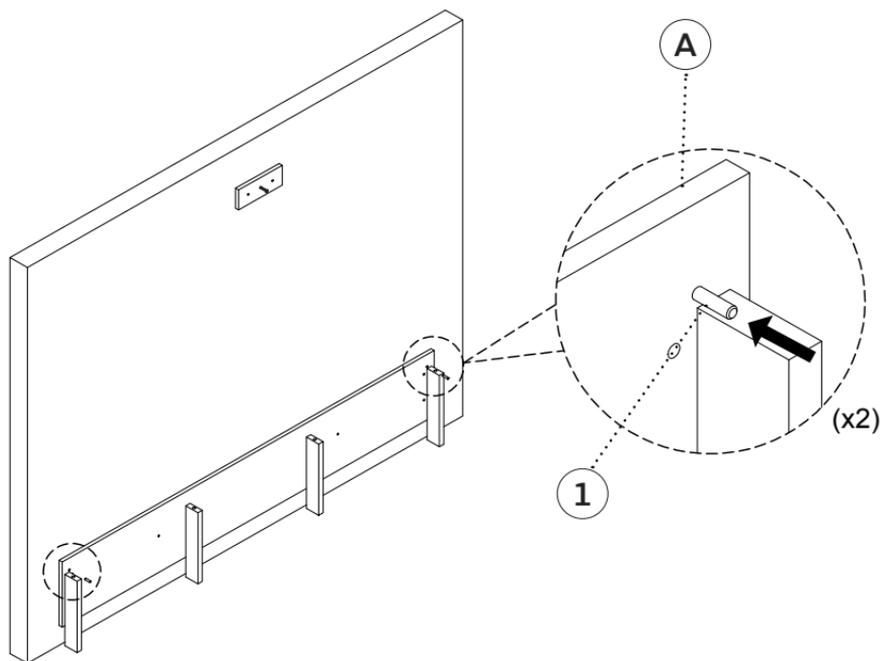


Step 6:

- A.** Utilizing two people, flip the Mattress (C) so the upholstery is facing up, and the legs should be facing forward, as pictured.
- B.** Align the wooden ledge on the Mattress (C) with the four legs mounted to the Mounting Board (A).
- C.** Locate four Bolts (2), thread a Spring Washer (3), and then a Flat Washer (4) onto all four of them.
- D.** Insert the four Bolts (2) through the wooden ledge and into the premounted legs to secure it in place.

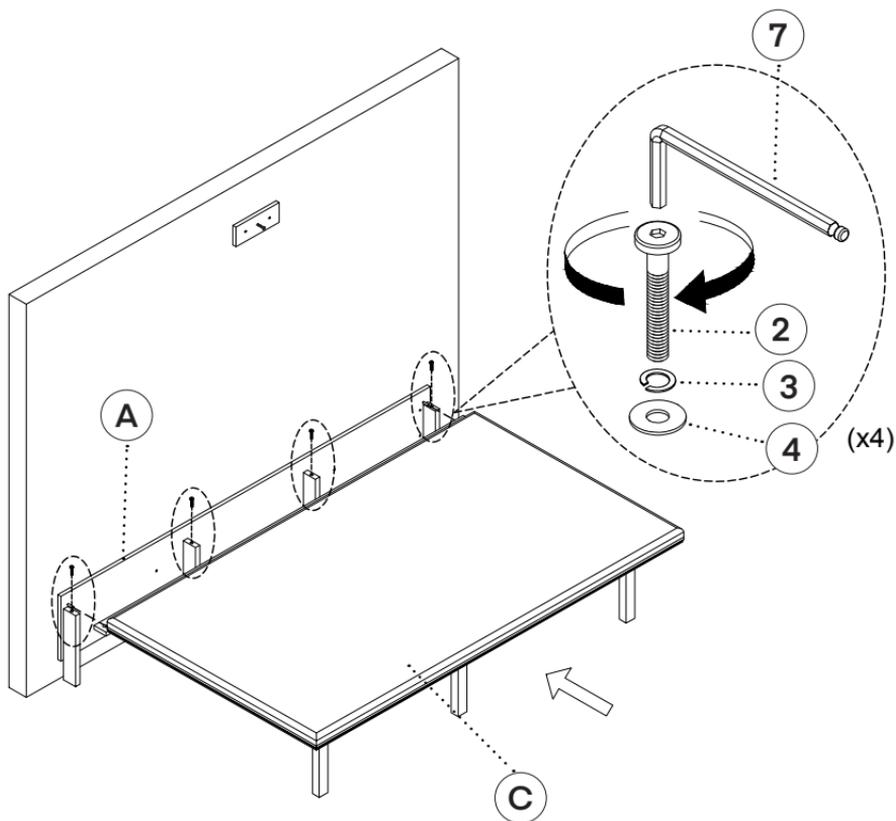

Step 7:

- A. Take the Chain (8) and clip it onto the preinstalled Bolt (2) on the Chain Block (B), as pictured.
- B. On the three forward-facing legs, adjust the height adjustment knobs on the bottom until the legs sit level.
- C. Your Wall-Mounted Folding Therapy Table is ready to be used.

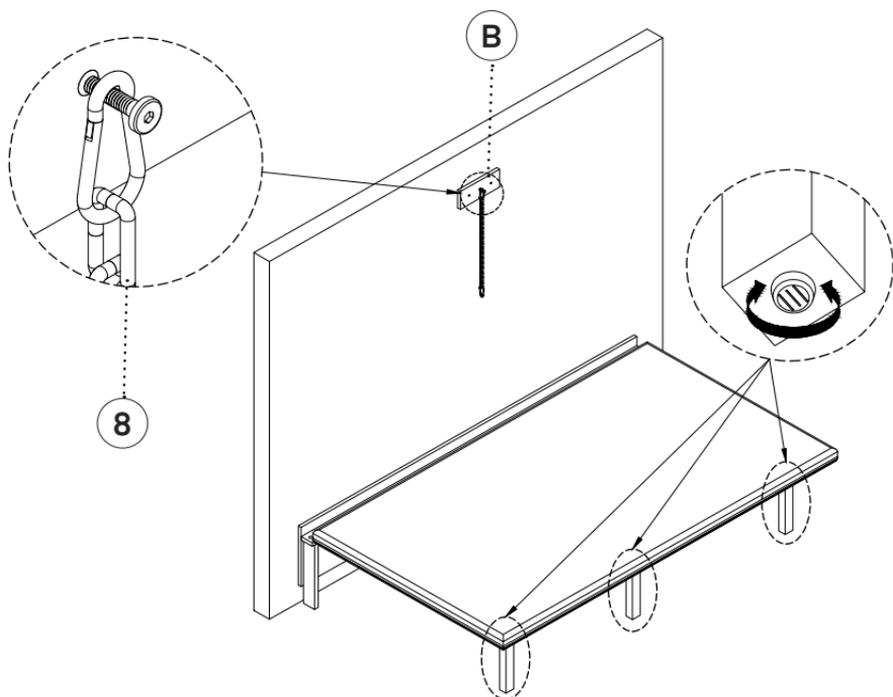


How To Fold and Mount

Step 1:

- A. When not in use, ensure all objects and people are clear of the table.
- B. Lift the Mattress (C) all the way up and clip the Chain (8) to the Bolt (2) that's preinstalled onto the Mattress (C), as pictured.


Step 2:

- A. Once the table is secured in place, depress the levers located behind the legs by pressing them toward you, as pictured.
- B. Proceed to fold in the three forward-facing legs.

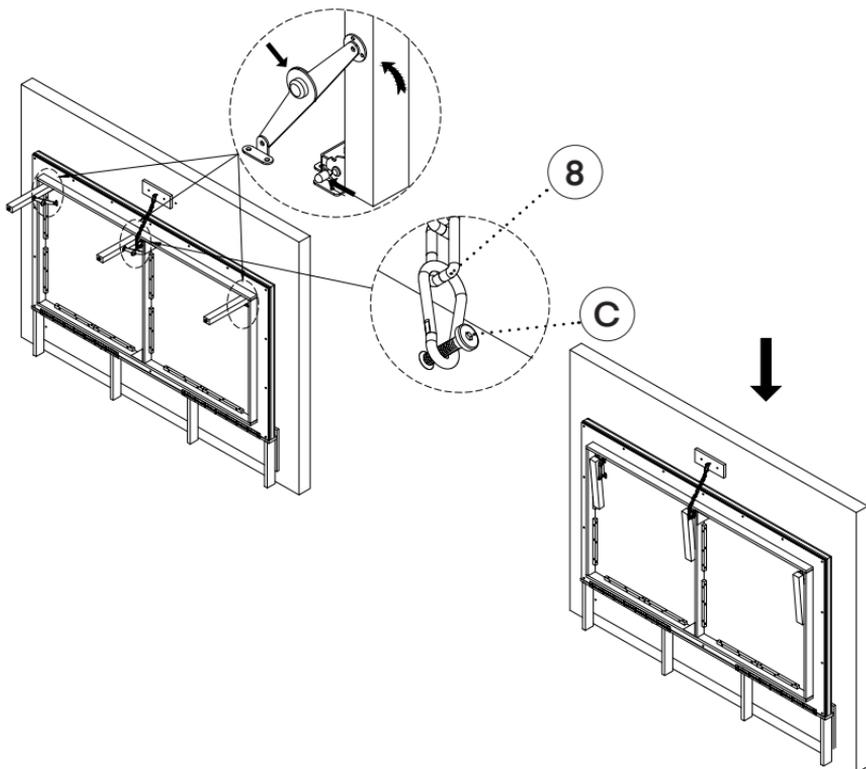

Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 1 year from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.

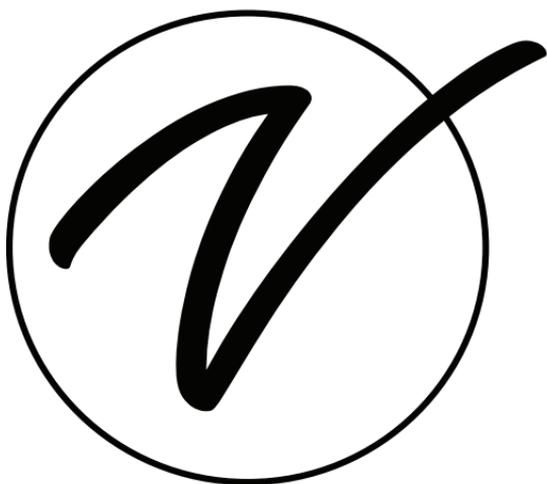

ADI996-06-SERIES

© 2025 Viva Comfort

vivacomfort.com