


ALTO TREATMENT TABLE WITH SHELF & ADJUSTABLE BACK

.....

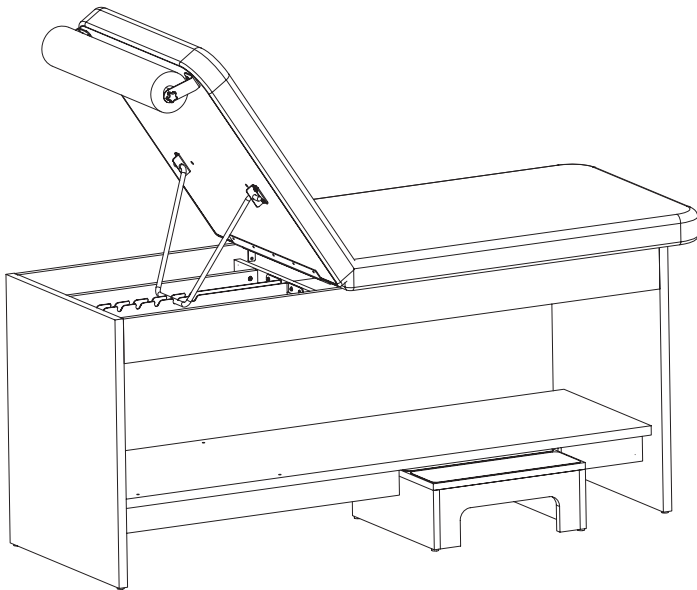

USER GUIDE

SKU: ADI996-14-SERIES



Thank you for your purchase.



Viva Comfort maintains a strong practice of ongoing development of products and commercial research.




We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.

Table of Contents

- 2 Parts List
- 3 Hardware List
- 4 Tools Needed
- 5 Specifications
- 6 Safety Warning
- 9 Assembly Instructions
- 16 Care & Maintenance Guide
- 17 Warranty
- 18 Notes


Parts List

A

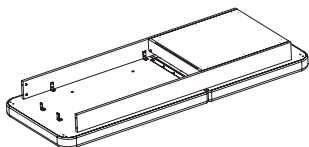


Upper Frame of Table (x1)

B

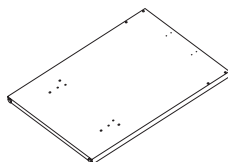


Panel Leg 1 (x1)

C

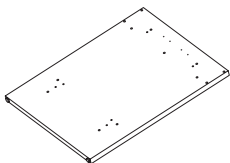


Panel Leg 2 (x1)

D

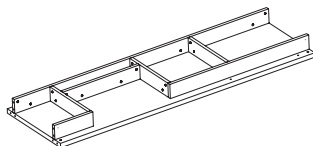


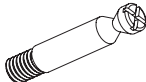



Lower Frame of Table (x1)

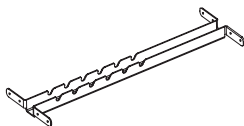E




Backrest
Adjustment Slot (x1)

Hardware List
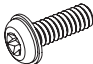
<p>1</p>  <p>Wooden Dowel (x10)</p>	<p>2</p>  <p>Cam Lock Dowel (x20)</p>
<p>3</p>  <p>Cam Lock Nut (x20)</p>	<p>4</p>  <p>Self Tapping Screw 5*18mm (x10)</p>
<p>5</p>  <p>Bolt 1/4*18mm (x4)</p>	

Tools Needed

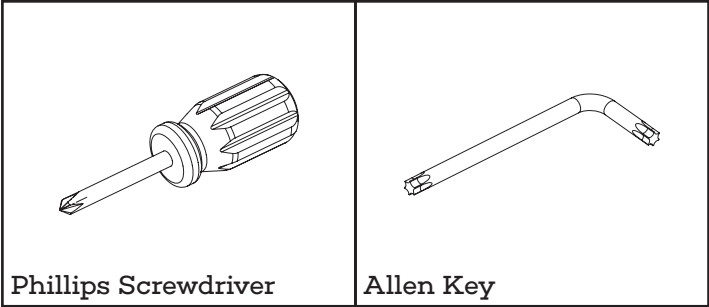

Specifications

Product Dimensions: 75" L x 32.7" W x 16" H

Weight Capacity: 137 lbs.

Recommended






Recommended Number of People Required for Assembly



2 persons

Safety Warning

The user is responsible for preventing damage to this product and injury to patients and/or users.

	Warning: Indication of potential personal injury.
	Warning: Be careful of pinching fingers.
	Reminder: Keep away from moisture. This device needs to be moisture and waterproof.
	Reminder: Always place the packaging box in the correct direction during transportation, storage, and handling to avoid damage and safety problems caused by incorrect placement.
	Reminder: This product is fragile and needs to be handled with care.



Risk of Collapse:

If the patient's weight exceeds the working load, the treatment table and the backrest may collapse if the patient's weight is not evenly distributed on the table surface.



Pinch Risk:

When the backrest moves down to the end, a hand or finger is placed in the gap between it and the table frame.

Infection risk:

When cleaning, please wear gloves, masks, and eye protection to prevent the cleaning spray from rebounding from the contaminated surface.

Risk of damage to the table surface:

- A.** The ambient temperature requirement for using and storing the equipment is: 0/32°F- 40/104°F. If the equipment is exposed to an environment exceeding the specified temperature for a long time, it may cause the risk of cracking and peeling of the leather surface.
- B.** The humidity requirement for equipment transportation, use and storage environment is: 20%~80%. If the equipment is exposed to an overly dry or humid environment for a long time, the leather surface may crack or peel.
- C.** The leather of the treatment table is strictly prohibited from contacting acidic or alkaline substances. If a non-neutral detergent is used to scrub the table surface, the leather surface may crack or peel prematurely, reducing its service life.
- D.** The leather of the table surface is prohibited from contacting liquid water. Liquid water may penetrate the leather, causing the sponge inside the leather to degenerate, affect its use, or cause the leather surface to crack or peel.

Hardware accessories rusting risk:

- A.** The humidity requirement for equipment transportation, use, and storage environment is 20%~80%. If the equipment is exposed to an overly humid environment for a long time, the hardware accessories may rust.
- B.** The metal parts of the equipment are strictly prohibited from contacting acidic substances; otherwise, they may cause damage to the paint surface and then rust.
- C.** The metal parts of the equipment are strictly prohibited from contacting liquid water, which may cause metal materials to rust.

Other precautions:

- A.** It is prohibited to place near any fire sources.
- B.** Do not place it in direct sunlight.
- C.** Do not place it in high-temperature places or near any heaters.
- D.** If the table legs are saturated in water or if the table is placed in a humid environment for a long time, it may cause the board to expand and the skin to peel off.

Patient Load Capacity: 500 lbs.

Instructions

Preparation and inspection before use:


- A.** Confirm whether a product manual is included.
- B.** Confirm whether the installation hardware package is attached.
- C.** Confirm whether the components are not deformed or damaged.
- D.** If any abnormalities occur during installation and use, please get in touch with the dealer in time.
- E.** Please remove the product from the packaging and place it on a level ground. Lay a blanket on the ground to prevent scratches on the board and leather, and ensure that the ground is clean and free of pollutants or sharp objects.

Assembly Instructions

Step 1:



- A.** Insert four Wooden Dowels (1) into the inner holes of Panel Leg 1 (B), as shown.
- B.** Insert four Wooden Dowels (1) into the inner holes of Panel Leg 2 (C), as shown.
- C.** Press each Wooden Dowel (1) firmly into place until fully seated. Ensure all dowels are straight and securely fitted, these will align and connect the Leg Panels to other boards in later steps.

Note: If any dowel feels loose, apply a small amount of wood glue before inserting it to create a stronger joint.

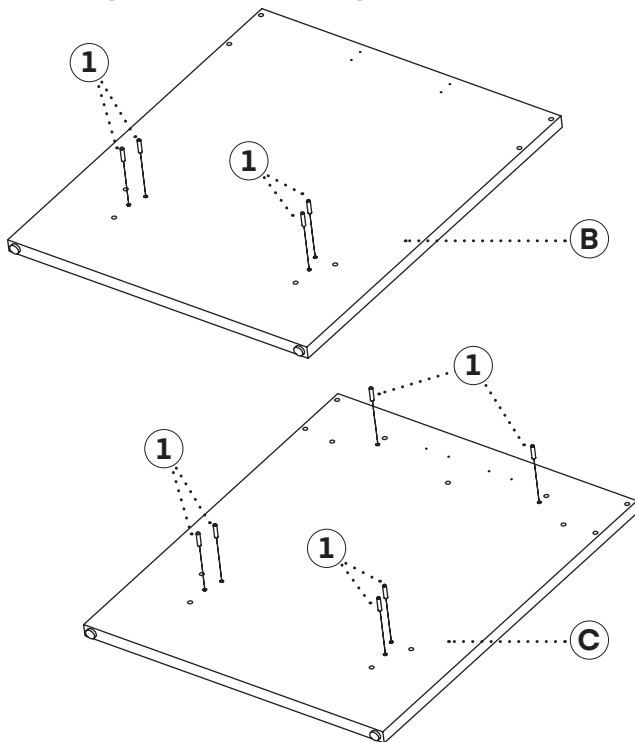

Step 2:


- A.** Insert eight Cam Lock Dowels (2) into the pre-drilled holes of Panel Leg 1 (B) and thirteen Cam Lock Dowels (2) into the pre-drilled holes of Panel Leg 2 (C).
- B.** Using a Phillip screwdriver, tighten each Cam Lock Dowel (2) until the shoulder of the dowel sits flush with the surface of the panel. Do not overtighten.
- a.** Ensure all dowels are inserted straight and securely fastened.

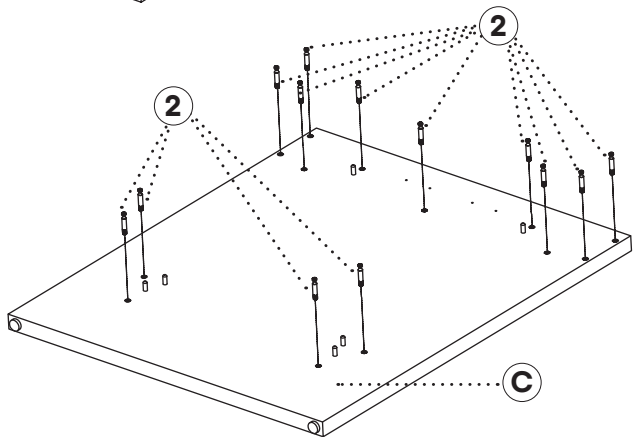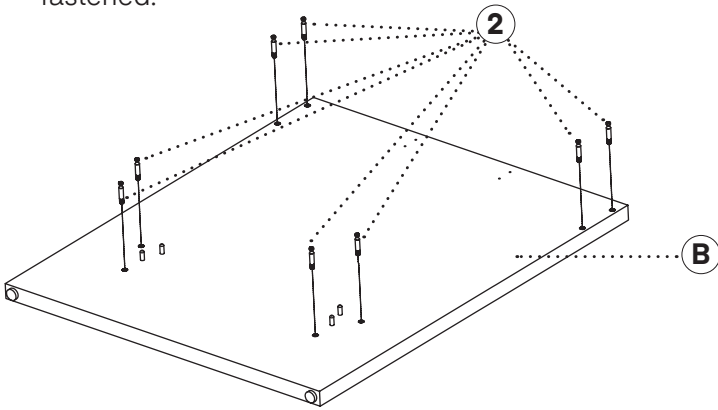

Step 3:


- A. Locate the Lower Frame (D) and determine the correct orientation of the reclining side and stool step opening according to your clinic's layout preference.
- B. Attach Left Leg Panel 1 (B) and Right Leg Panel 2 (C) to both the Upper Frame of the Table (A) and Lower Frame of the Table (D) using the previously installed Cam Lock Dowels (2).
- C. If necessary, adjust the direction of the Lower Frame (D) before fully tightening the cam locks.

Note: Do not overtighten the cam locks. Stop turning once you feel resistance.

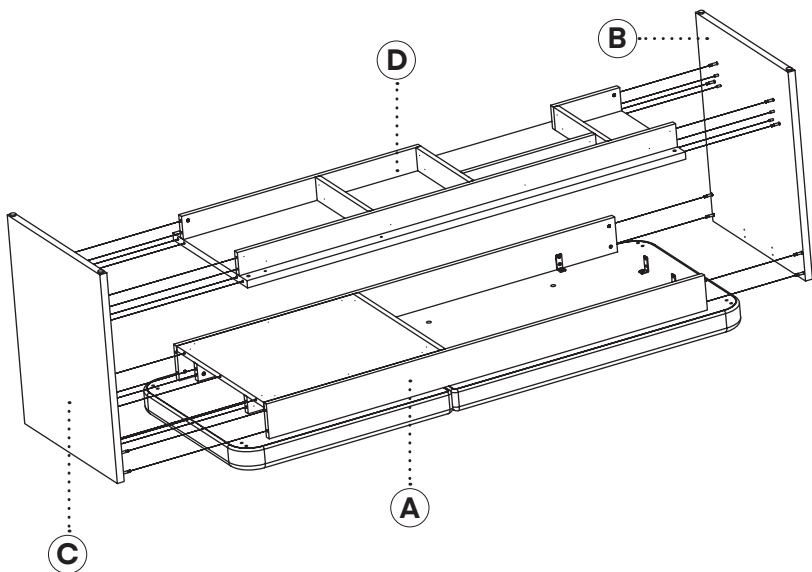

Step 4:


- A.** Once the panels are properly aligned, insert all Cam Lock Nuts (3) into the designated holes. Secure each by turning clockwise with a Phillips screwdriver, as shown.
- B.** Ensure all Cam Lock Nuts (3) are fully tightened and that the overall structure is stable before proceeding.

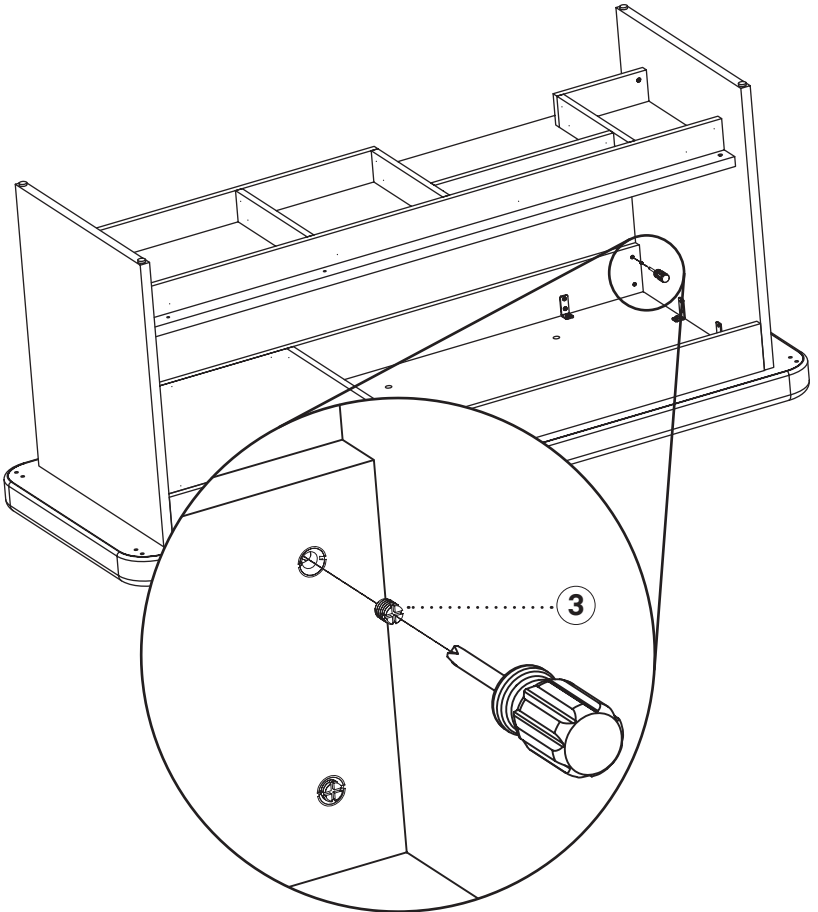

Step 5:


- A.** Using a Phillips screwdriver, fasten the L-shaped metal brackets located at the bottom of the Upper Frame of the Table (A) to the inner sides of Leg Panel 1 (B) and Leg Panel 2 (C).
- B.** Insert and tighten the Self-Tapping Screws (4) through the holes in each bracket and into the pre-drilled pilot holes on the Leg Panels.
- C.** Ensure each bracket sits flush against both surfaces and that all screws are securely tightened to create a stable connection.

Note: The Leg Panels (B/C) are pre-drilled for screw installation. Do not overtighten, as this may damage the surface of the panels.

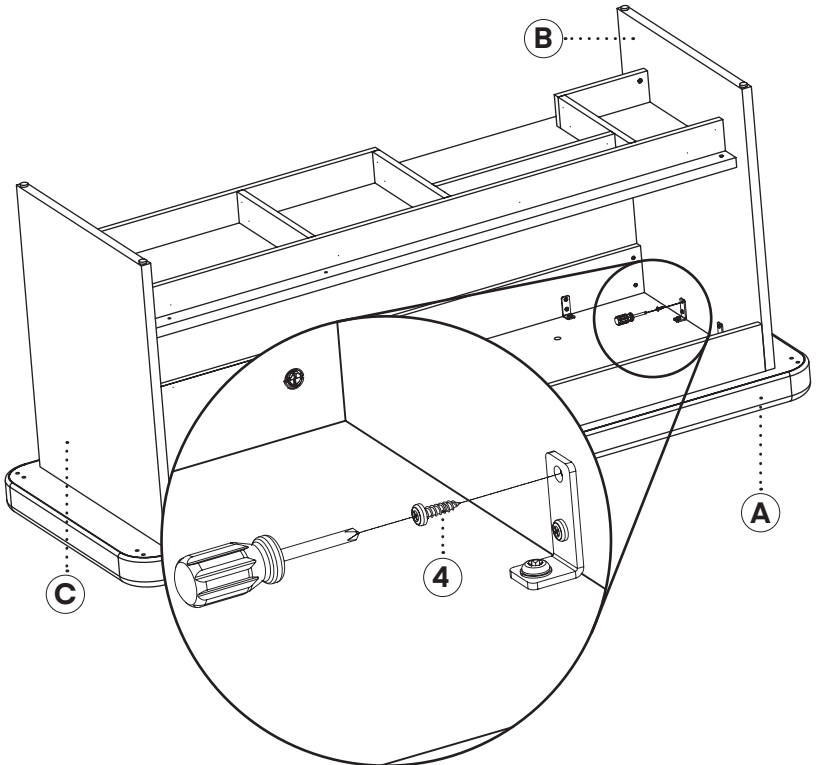

Step 6:


- A. Utilizing two people, carefully turn the Table over so that the upholstery faces upward and the legs face downward.
- B. Position the Backrest Adjustment Slot (E) on the reclining side of the Upper Frame of the Table (A), as shown.
- C. Align the holes of the Backrest Adjustment Slot (E) with the pre-drilled holes on the Upper Frame (A).
- D. Using a Phillips screwdriver, fasten the Backrest Adjustment Slot (E) securely with four Self-Tapping Screws (4).
- E. Confirm that the slot is firmly attached and moves smoothly without obstruction.

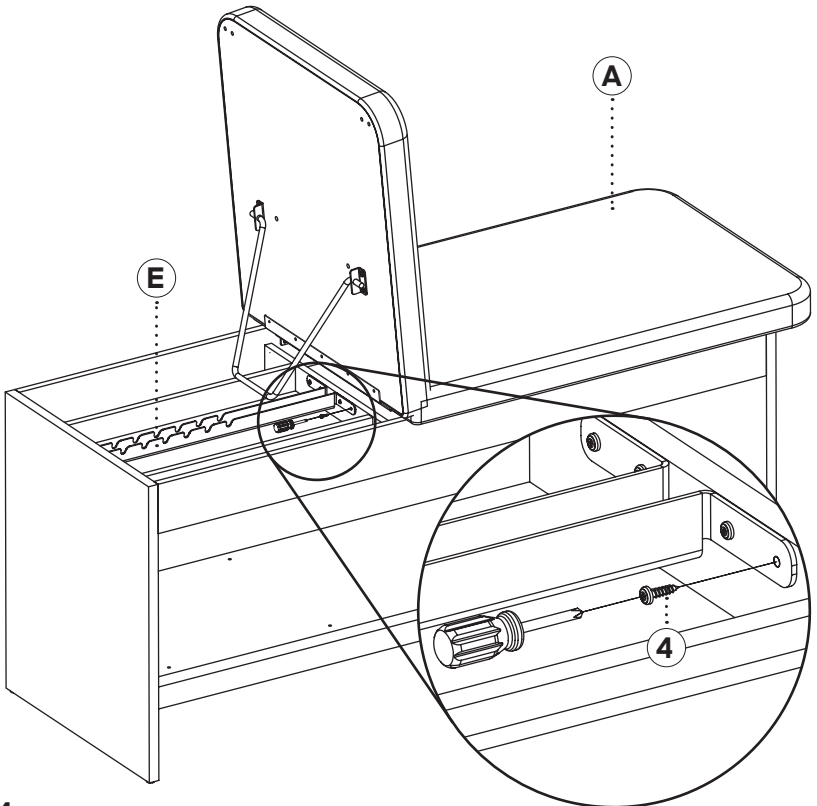

Adjusting the Backrest Angle:

- A. Gently lift the backrest upward by hand, then lower the support arm beneath it so that the lower end of the support arm fits securely into the slot on the table's frame.
- B. To return the backrest to a flat position, perform the steps in reverse order.

Note: Do not adjust the backrest angle while a patient is leaning on it. If the support arm is not secured in the slot and the backrest bears weight, it may fall suddenly, risking injury such as pinching the operator's arm.

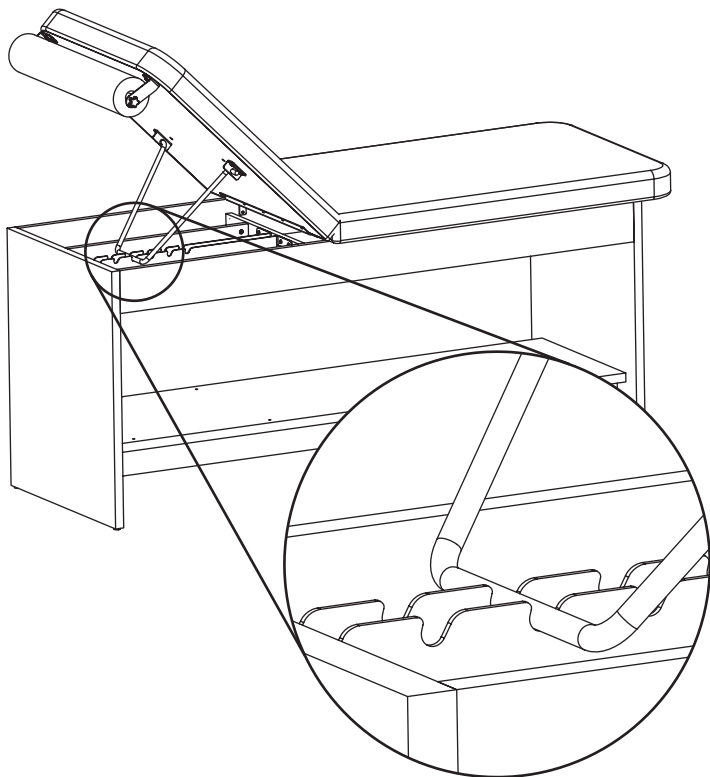

Leather Maintenance

- A.** It is recommended to use a small amount of neutral detergent/cleaner or wipe with a dry towel when wiping the leather, then wipe it clean with a dry cloth, dry it, and place it in a ventilated place to dry.
- B.** If disinfection is necessary, it is acceptable to wipe it with hypochlorous acid disinfectant (HClO) or alcohol with a concentration below 70%. Highly corrosive disinfectants are prohibited for disinfection; otherwise, they may accelerate the aging of the leather, causing the leather to change color, deform, harden, become brittle, or even peel off the surface.
- C.** When not used for a long time, it is recommended that the table be covered with a table cover and stored in a dry environment. Avoid direct sunlight, rain, humidity, and other environments.
- D.** Too high or too low-temperature environments may accelerate the aging of leather.
- E.** Sharp objects are prohibited from touching the leather surface.


Maintenance of Metal and Plates

- A.** A dry towel is recommended to wipe metal and plates.
- B.** If disinfection is necessary, it is acceptable to use hypochlorous acid disinfectant or alcohol with a concentration below 70% to the maximum extent possible. Using highly corrosive disinfectants is prohibited for disinfection. Otherwise, the paint or wood veneer may be damaged.

Maintenance execution items	Minimum frequency
Test the stability of the Table frame when pushed horizontally/vertically/laterally. If shaking occurs, tighten the connecting nuts.	Quarterly

Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 1 year from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.

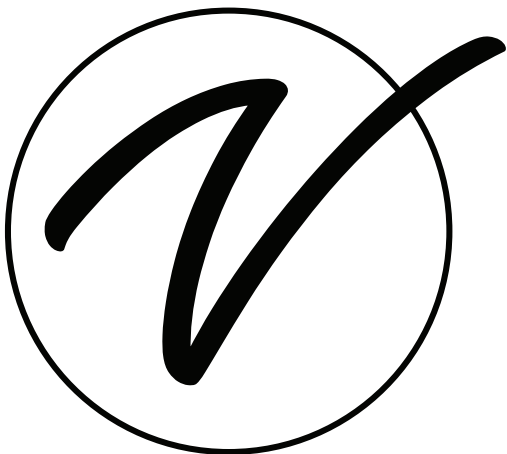

SKU: ADI996-14-SERIES

© 2025 Viva Comfort

vivacomfort.com