



.....

# REHABILITATION STAIRCASE

## DOUBLE-SIDE

.....

W19286

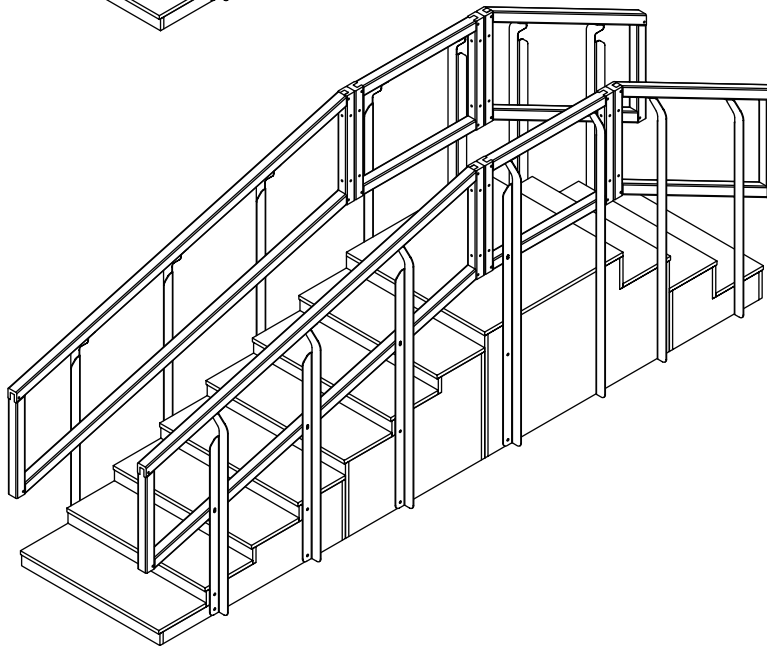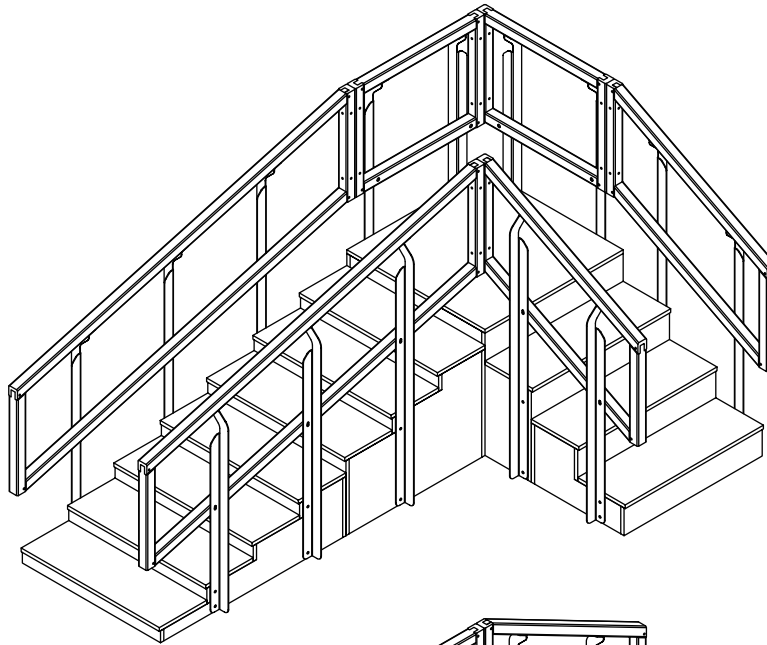

**USER GUIDE**

**SKU: ADIME906-02-30**



# **Thank you for your purchase.**



**Viva Comfort maintains a strong practice of ongoing development of products and commercial research.**















**We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.**

# Table of Contents
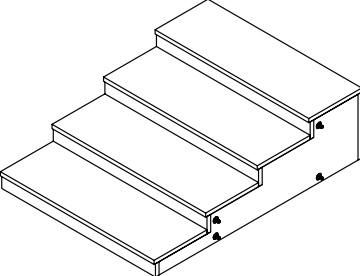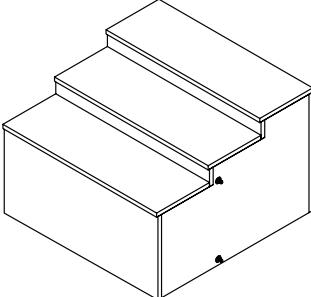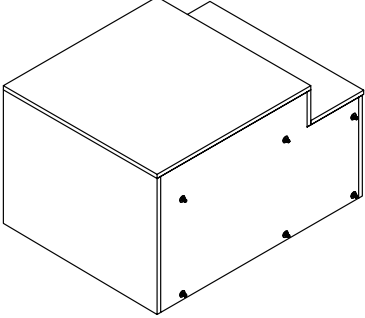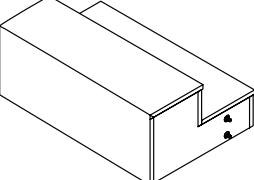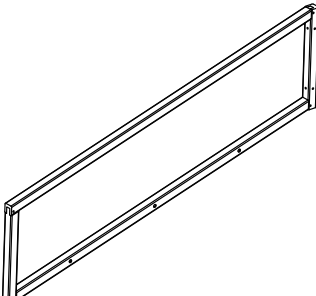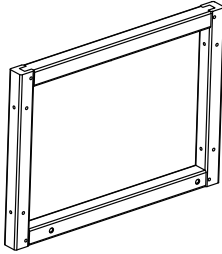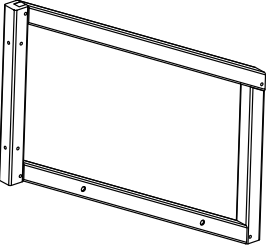
<b>2</b>	Parts List
<b>3</b>	Hardware List
<b>4</b>	Specifications
<b>4</b>	Recommendation
<b>4</b>	Preparation and Inspection Before Use
<b>5</b>	Three Configurations Assembly Instructions
<b>6</b>	Straight Line
<b>10</b>	Right L-Shape
<b>14</b>	Left L-Shape
<b>18</b>	Safety Warning
<b>19</b>	Care & Maintenance Guide
<b>20</b>	Warranty
<b>21</b>	Notes

# Parts List

<p><b>A</b></p>  <p>Stair Section 1</p>	<p><b>B</b></p>  <p>Stair Section 2</p>	<p><b>C</b></p>  <p>Stair Section 3</p>
<p><b>D</b></p>  <p>Stair Section 4</p>	<p><b>E</b></p>  <p>Guardrail 1 (x2)</p>	<p><b>F</b></p>  <p>Guardrail 2 (x2)</p>
<p><b>G</b></p>  <p>Guardrail 3 (x2)</p>	<p><b>H</b></p>  <p>Handrail Metal Rod Left</p>	<p><b>H1</b></p>  <p>Handrail Metal Rod Right</p>
<p><b>I</b></p>  <p>Handrail Metal Rod Left (x2)</p>	<p><b>I1</b></p>  <p>Handrail Metal Rod Right (x2)</p>	<p><b>J</b></p>  <p>Handrail Metal Rod Left (x2)</p>

**J1**

Handrail Metal Rod Right  
(x2)


**K**

Handrail Metal Rod Left (x2)


**K1**

Handrail Metal Rod Right  
(x2)


## Hardware List

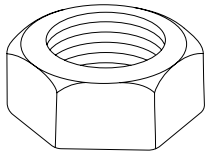**1**


M10 Lock Nut (x40)

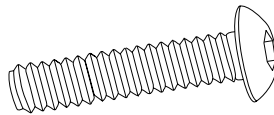**2**


Hex Head Bolt (x8)  
(M8\*120mm)

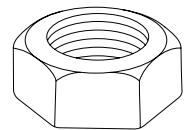**3**


M8 Lock Nut (x8)

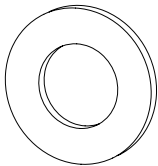**4**


Bolt Washer (x16)  
( $\phi 16 * \phi 9 * 1$ )

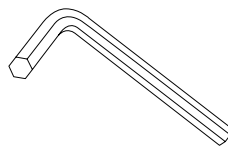**5**


Hex Wrench

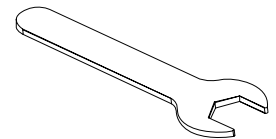**6**


Open-End Wrench M10

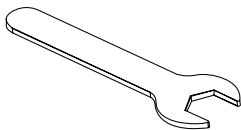**7**


Open-End Wrench M8

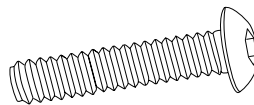**8**


Hex Head Bolt (x28)  
(M8\*20mm)

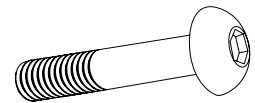**9**

Hex Head Bolt (x12)  
(M10\*55mm)

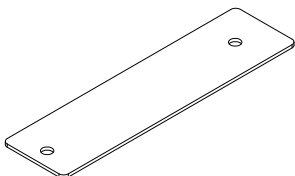**10**

Short Steel Plate (x8)

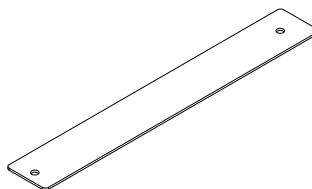**11**

Long Steel Plate (x4)

# Specifications

**Product Dimensions:** "L x " W x " H

**Weight Capacity:** 207 kg. / 500 lbs.

## Recommendation

Recommended Number of People Required for Assembly




**3 persons**

## Preparation and Inspection Before Use


- A.** Confirm whether a product manual and warranty card accompany the product.
- B.** Check whether the installation hardware package is attached.
- C.** Check whether the components are deformed or damaged.
- D.** If any abnormalities occur during installation and use, please get in touch with the dealer in time.

# Three Configurations


There are **three installation methods** for this staircase: Straight Line, Left L-Shape, and Right L-Shape. Please find the installation method you want, the relevant accessories and their correct installation positions as shown.

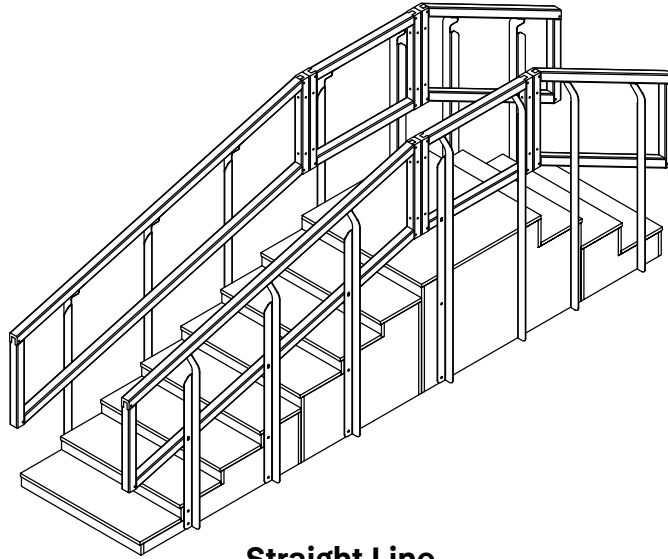

**Straight Line**

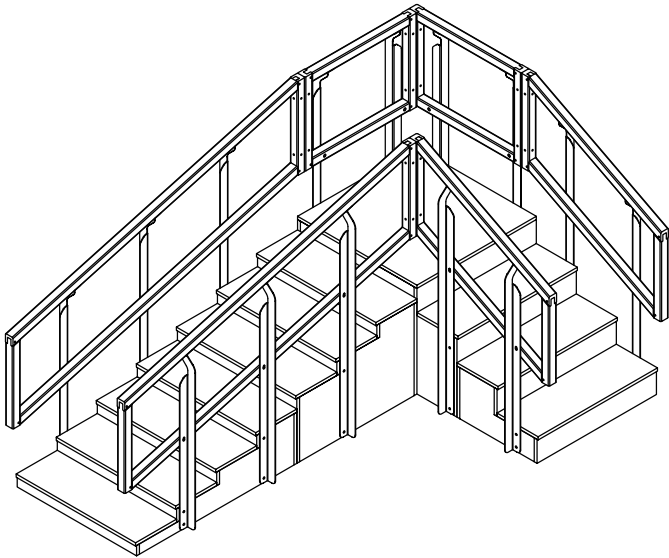

**Right L-Shape**

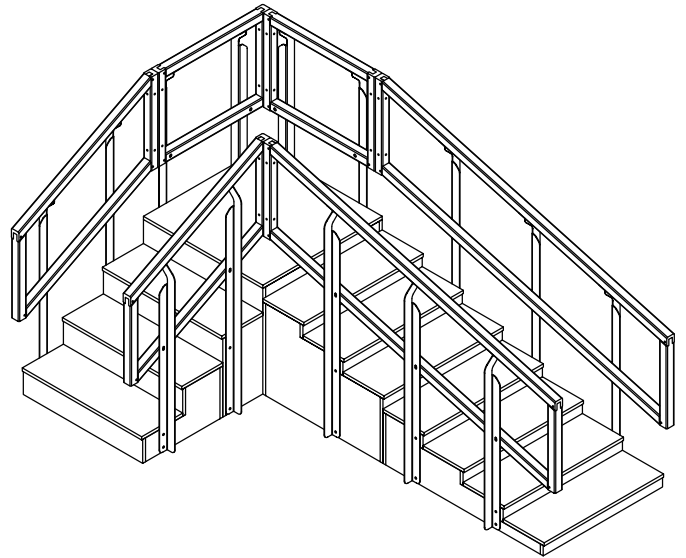

**Left L-Shape**

# Assembly Instructions


## Straight Line

### Step 1:

- A. Remove the product from the packaging and place it on a flat, horizontal surface with the steps facing upward.
- a. It is recommended to lay a blanket or protective covering on the ground to prevent damage to the floor.

### Step 2:

- A. Connect and assemble the four Stair Sections (A), (B), (C), and (D), as shown in the diagram (Bottom View).
- B. Locate the Short Steel Plates (10), Long Steel Plates (11), and M10x55 Bolts (9).

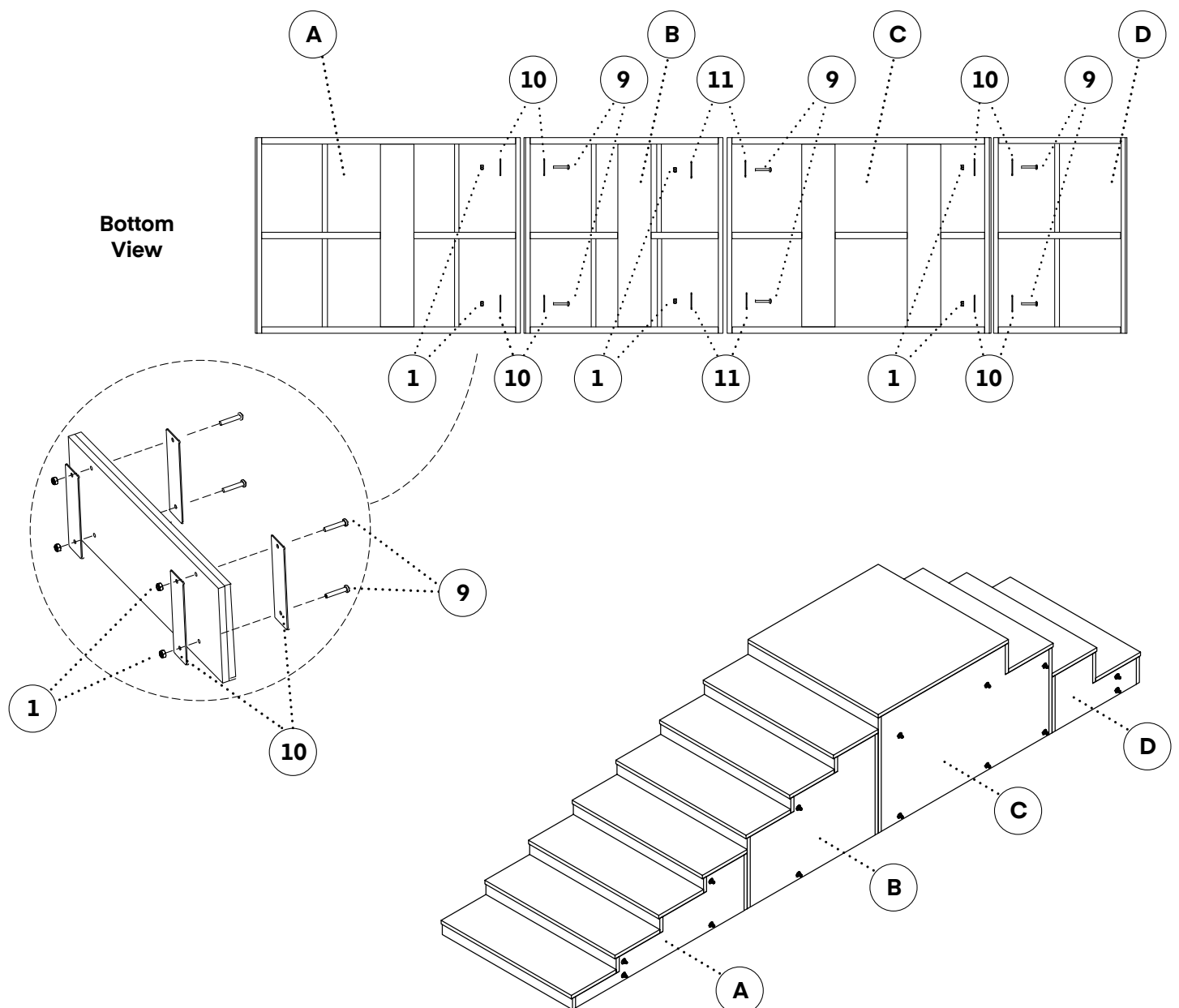

## Step 2:


- A.** Use the M8x120 Bolts (2) to assemble the Guardrails (E), (F), (G). Insert each Bolt (2) through a Washer (4), the pre-drilled holes on the side of the guardrail, another Washer (4), and then secure it with a M8 (3) nut using the Open-end Wrench (6).
- B.** Install three Guardrails (E), (F), (G) on the left side and three on the right side.

### Hardware required (total):

8 × M8x120 Bolts (2)

16 × Washers (4)

8 × M8 Nuts (3)

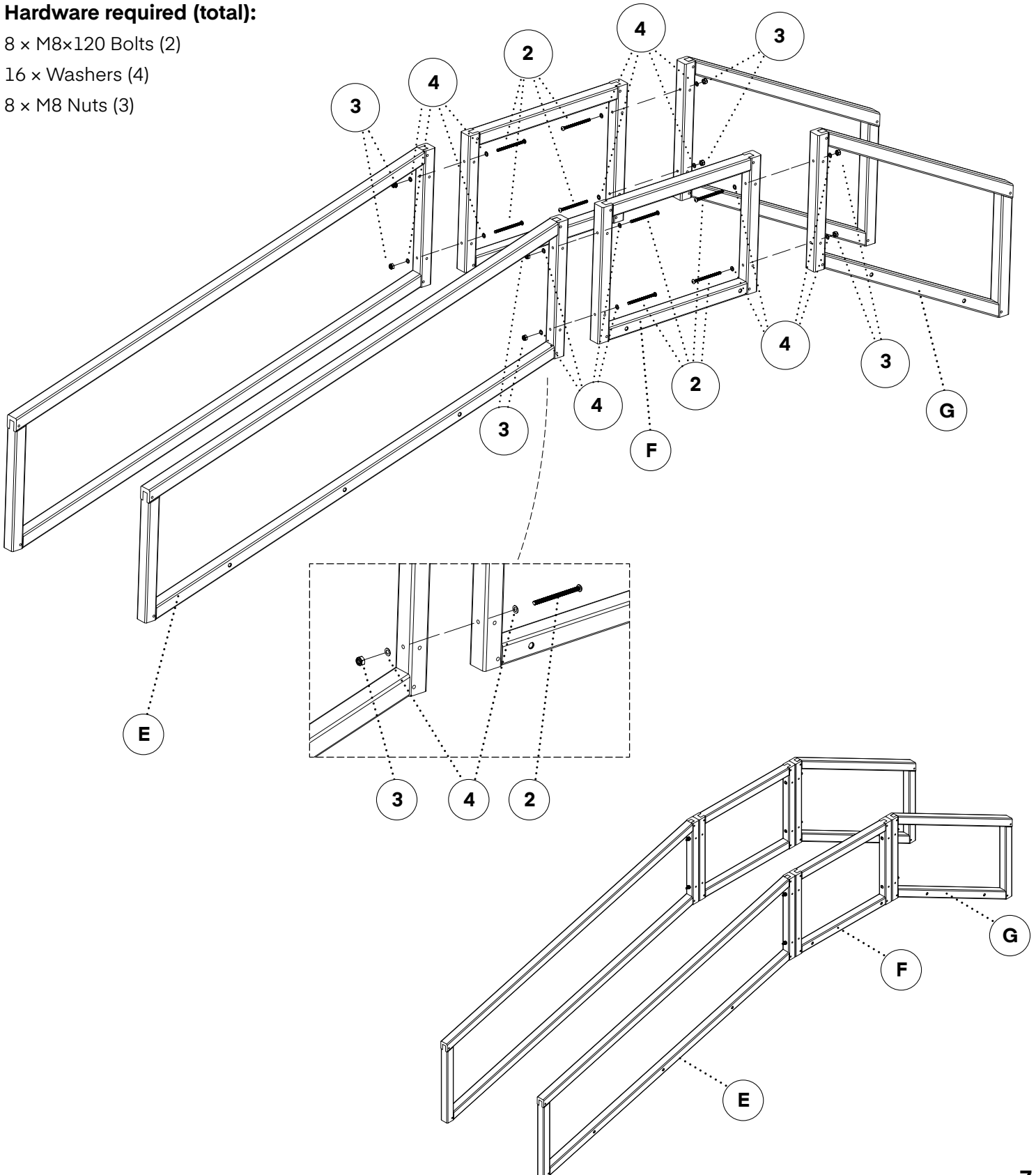

**Step 3:**


**A.** Align the pre-drilled hole at the bottom of each Handrail Metal Rod, pictured below, with the pre-installed bolt on the side of the stairs.


**Step 4:**

**A.** Secure each connection using a M10 Locking Nut (1) and tighten with a M10 Open-end Wrench (8).

**Hardware required (total):**

28 × M10 Locking Nuts (1)

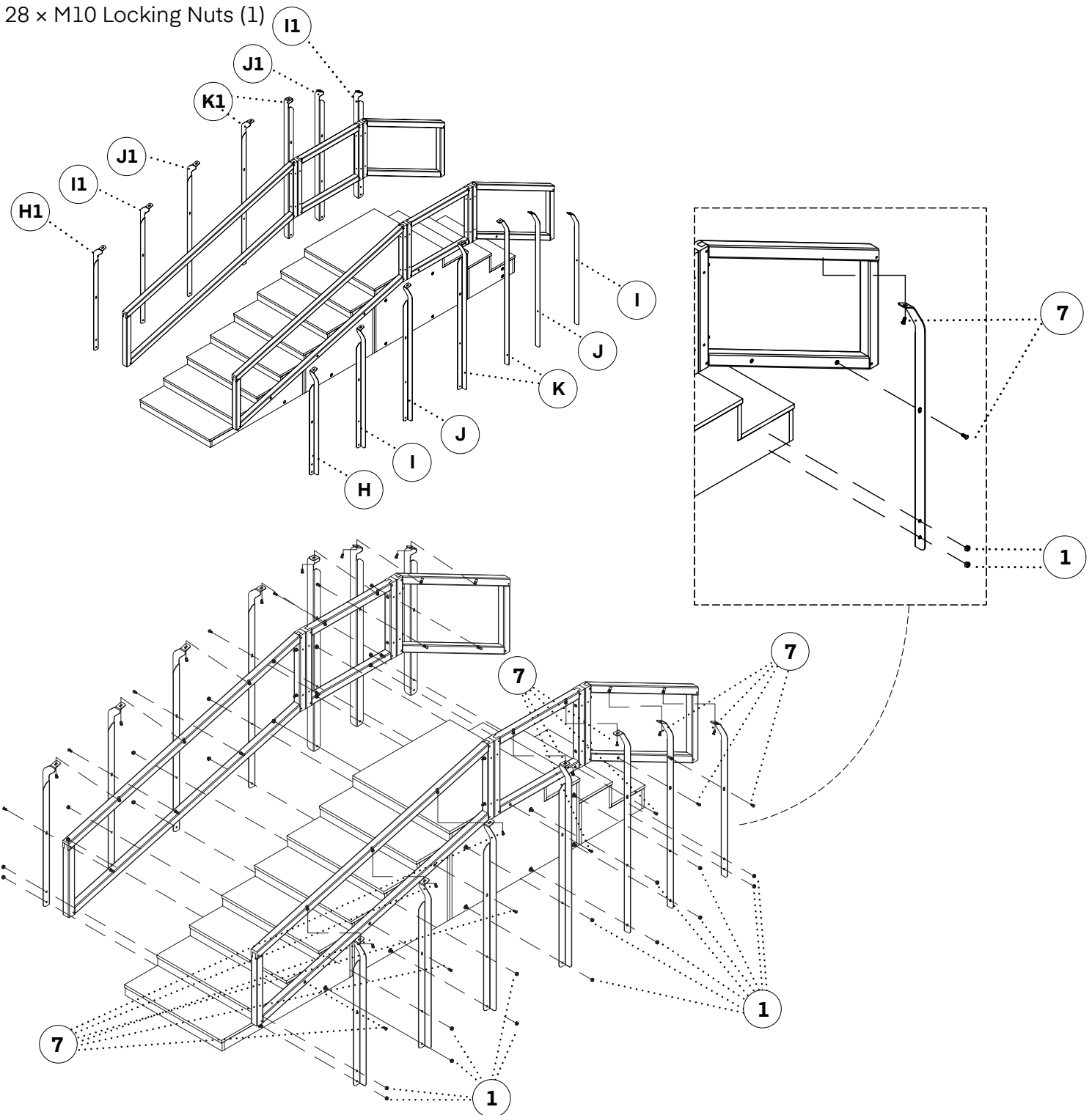

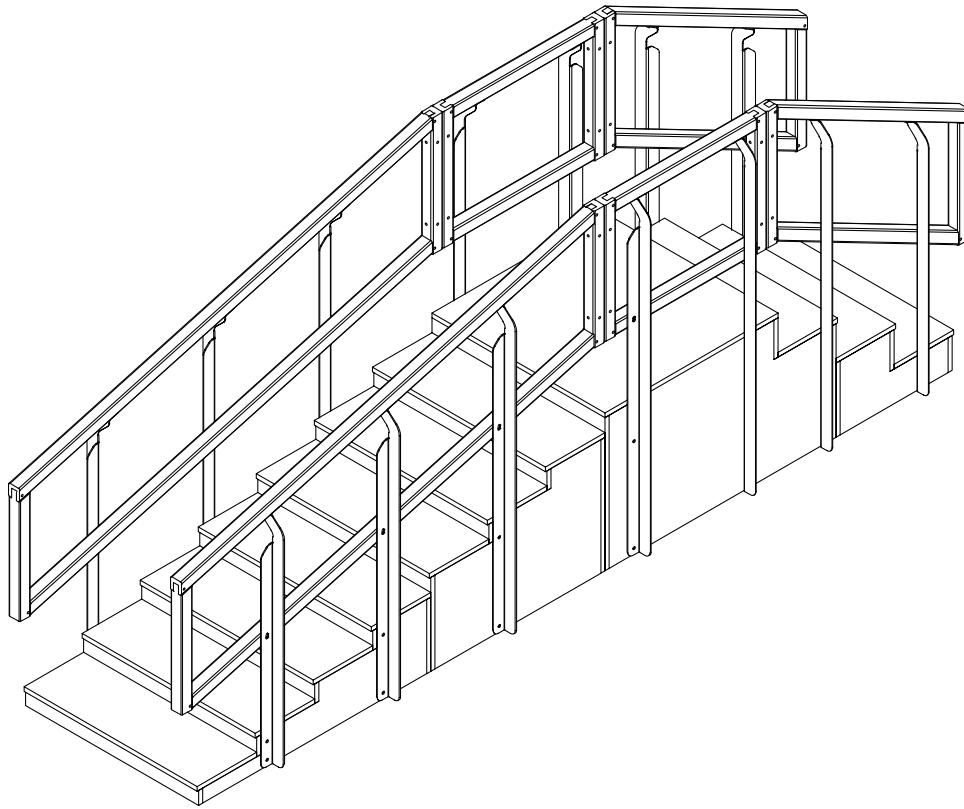

- ⚠ After initially locking all bolts, each bolt and nut must be tightened again to ensure they are secure. Check the guardrail for any movement—if it shakes, continue tightening all bolts and nuts until the guardrail is stable.
- ⚠ If the handrail remains unstable or shakes, discontinue use immediately. Failure to do so may result in injury or damage, for which the user will be solely responsible.

# Assembly Instructions


## Right L-Shape

### Step 1:

- A. Remove the product from the packaging and place it on a flat, horizontal surface with the steps facing upward.
- a. It is recommended to lay a blanket or protective covering on the ground to prevent damage to the floor.

### Step 2:

- A. Connect and assemble the four Stair Sections (A), (B), (C), and (D), as shown in the diagram (Bottom View).
- B. Locate the Short Steel Plates (10), Long Steel Plates (11), and M10x55 Bolts (9).

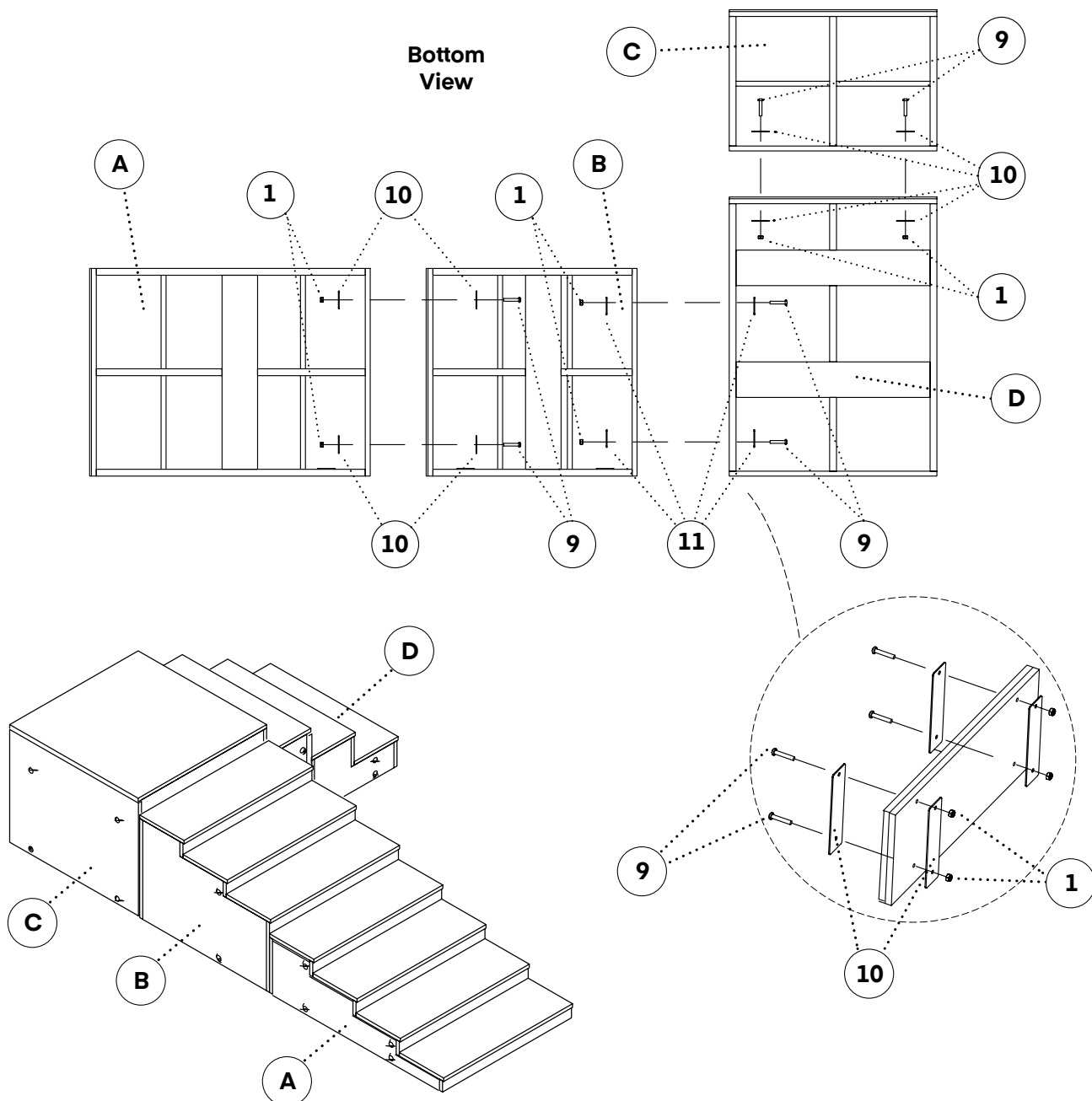

## Step 2:


- A.** Use the M8x120 Bolts (2) to assemble the Guardrails (E), (F), (G). Insert each Bolt (2) through a Washer (4), the pre-drilled holes on the side of the guardrail, another Washer (4), and then secure it with a M8 (3) nut using the Open-end Wrench (6).
- B.** Install the two Guardrails 1 (E) on the left side, two Guardrails 2 (F) at the center back, positioned side by side and perpendicular to each other, and two Guardrails 3 (G) on the right side.

### Hardware required (total):

8 × M8x120 Bolts (2)

16 × Washers (4)

8 × M8 Nuts (3)

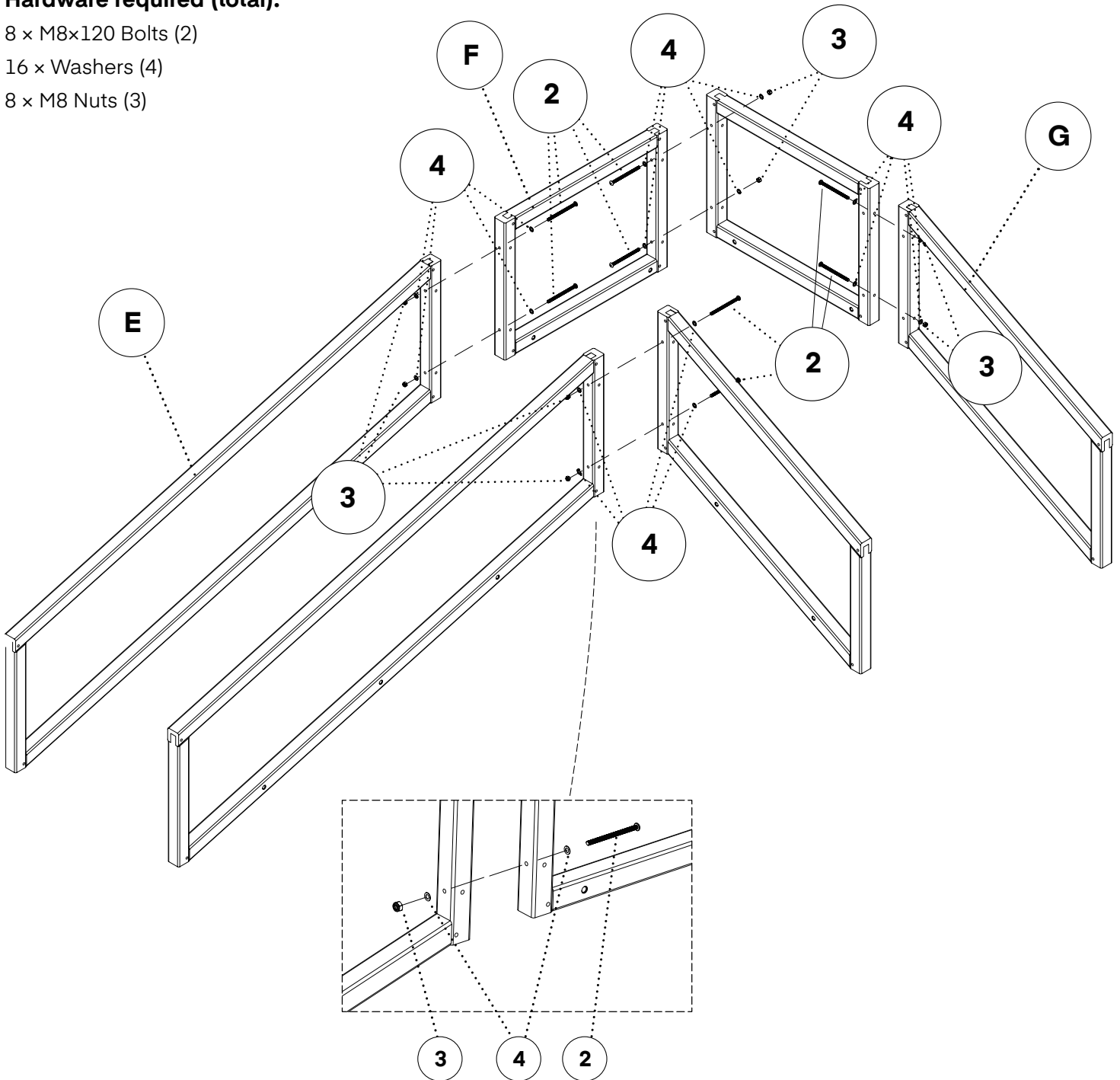

### Step 3:


A. Align the pre-drilled hole at the bottom of each Handrail Metal Rod, pictured below, with the pre-installed bolt on the side of the stairs.


### Step 4:

A. Secure each connection using a M10 Locking Nut (1) and tighten with a M10 Open-end Wrench (8).

### Hardware required (total):

28 x M10 Locking Nuts (1)

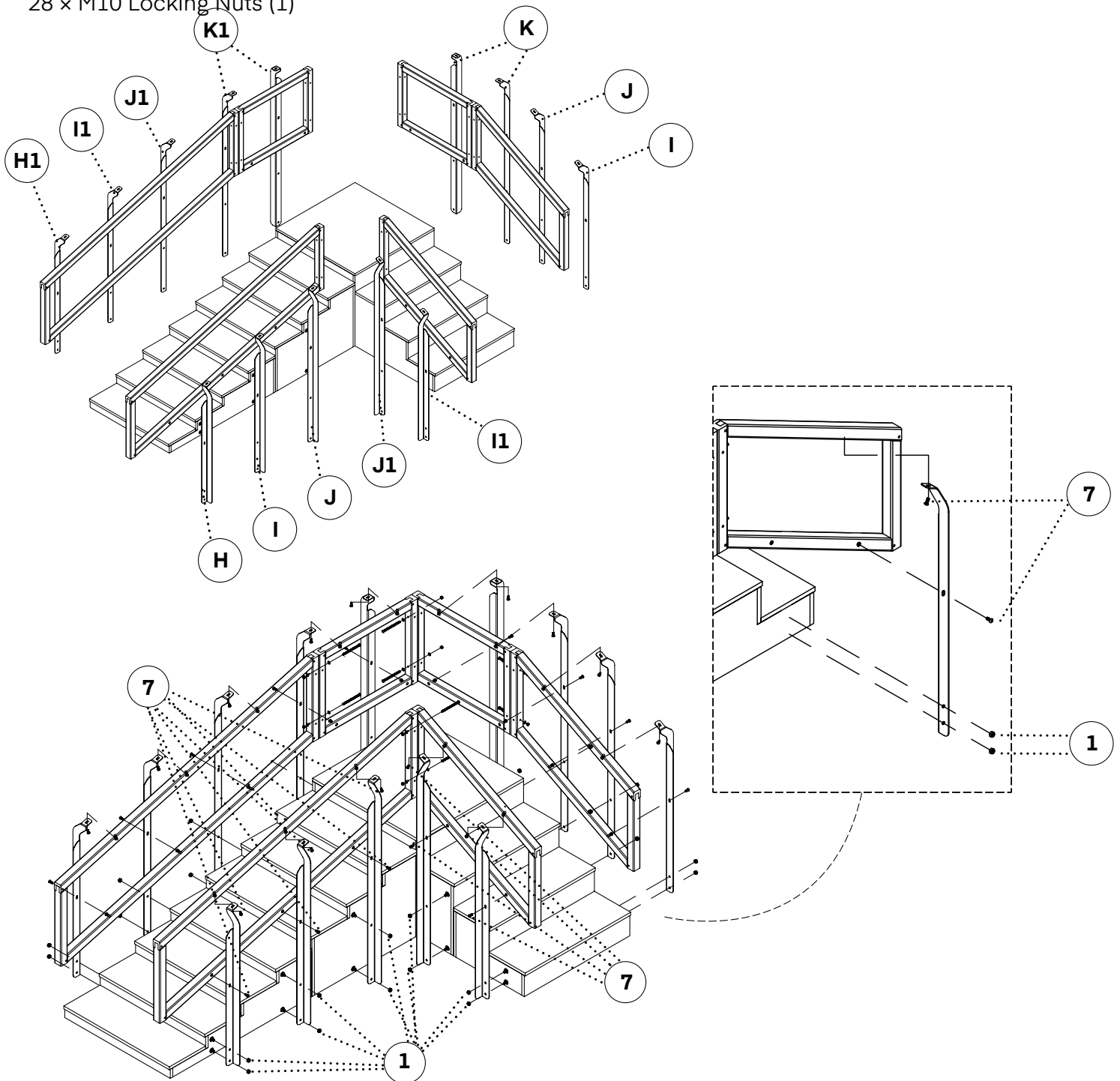

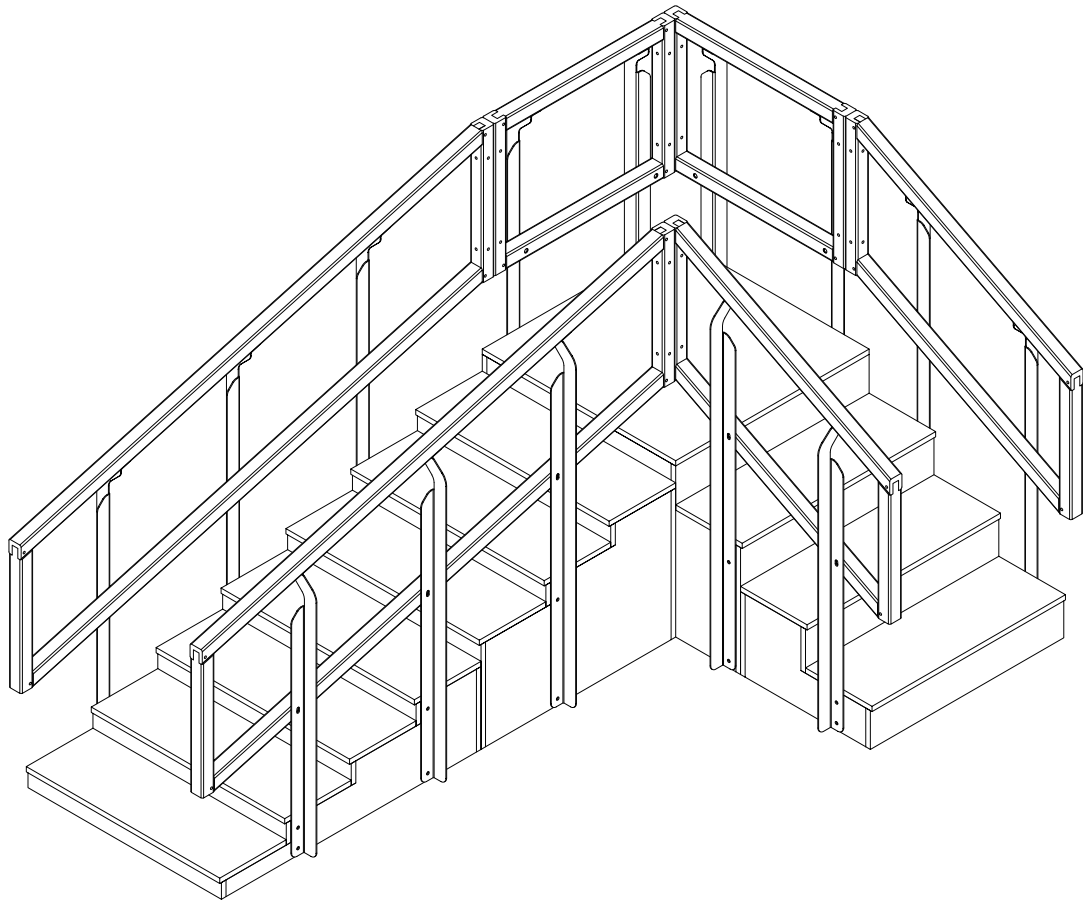

- ⚠ After initially locking all bolts, each bolt and nut must be tightened again to ensure they are secure. Check the guardrail for any movement—if it shakes, continue tightening all bolts and nuts until the guardrail is stable.
- ⚠ If the handrail remains unstable or shakes, discontinue use immediately. Failure to do so may result in injury or damage, for which the user will be solely responsible.

# Assembly Instructions


## Left L-Shape

### Step 1:

- A. Remove the product from the packaging and place it on a flat, horizontal surface with the steps facing upward.
- a. It is recommended to lay a blanket or protective covering on the ground to prevent damage to the floor.

### Step 2:

- A. Connect and assemble the four Stair Sections (A), (B), (C), and (D), as shown in the diagram (Bottom View).
- B. Locate the Short Steel Plates (10), Long Steel Plates (11), and M10x55 Bolts (9).

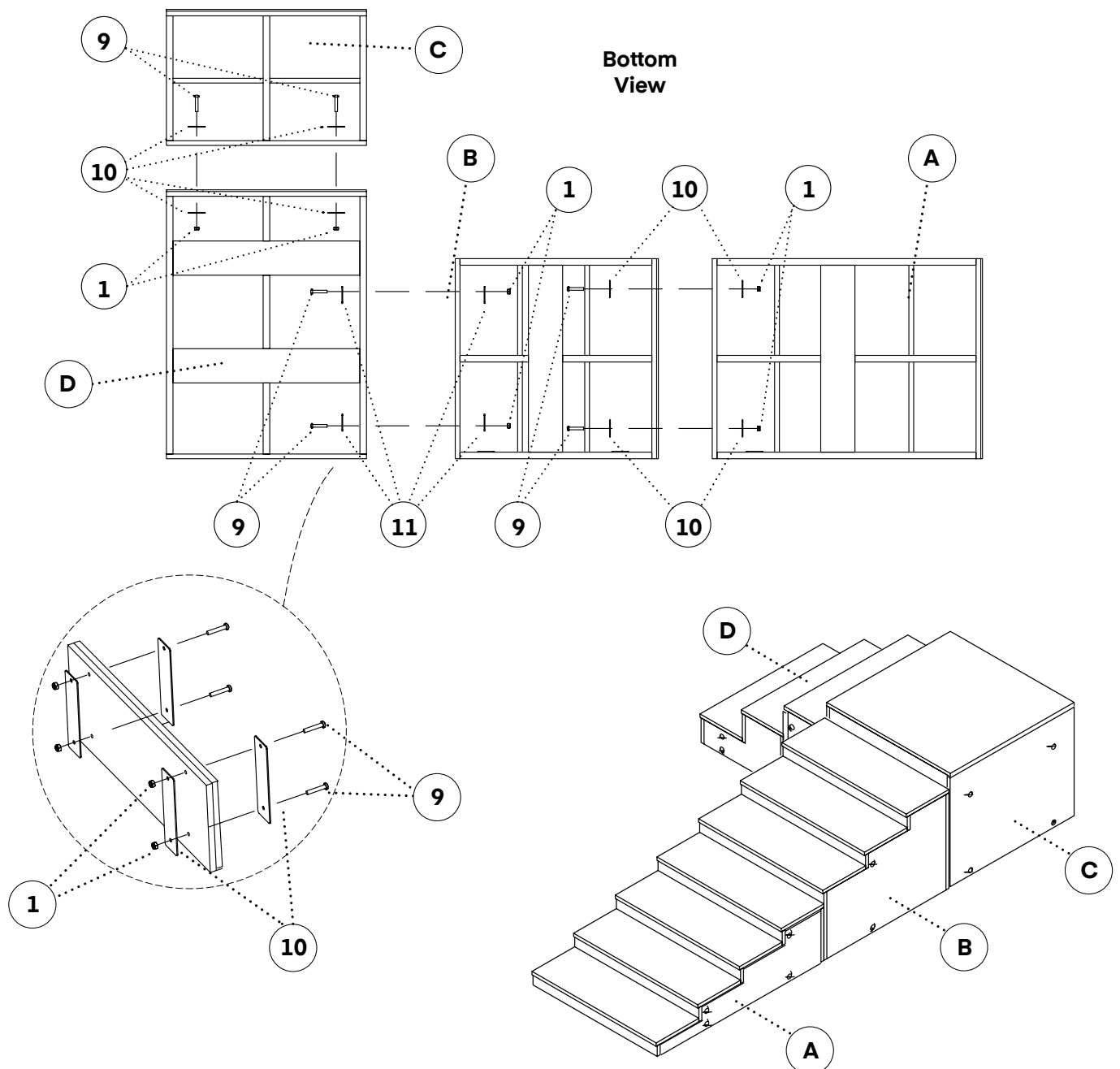

## Step 2:


- A.** Use the M8x120 Bolts (2) to assemble the Guardrails (E), (F), (G). Insert each Bolt (2) through a Washer (4), the pre-drilled holes on the side of the guardrail, another Washer (4), and then secure it with a M8 (3) nut using the Open-end Wrench (6).
- B.** Install the two Guardrails 1 (E) on the right side, two Guardrails 2 (F) at the center back, positioned side by side and perpendicular to each other, and two Guardrails 3 (G) on the left side.

### Hardware required (total):

8 × M8x120 Bolts (2)

16 × Washers (4)

8 × M8 Nuts (3)

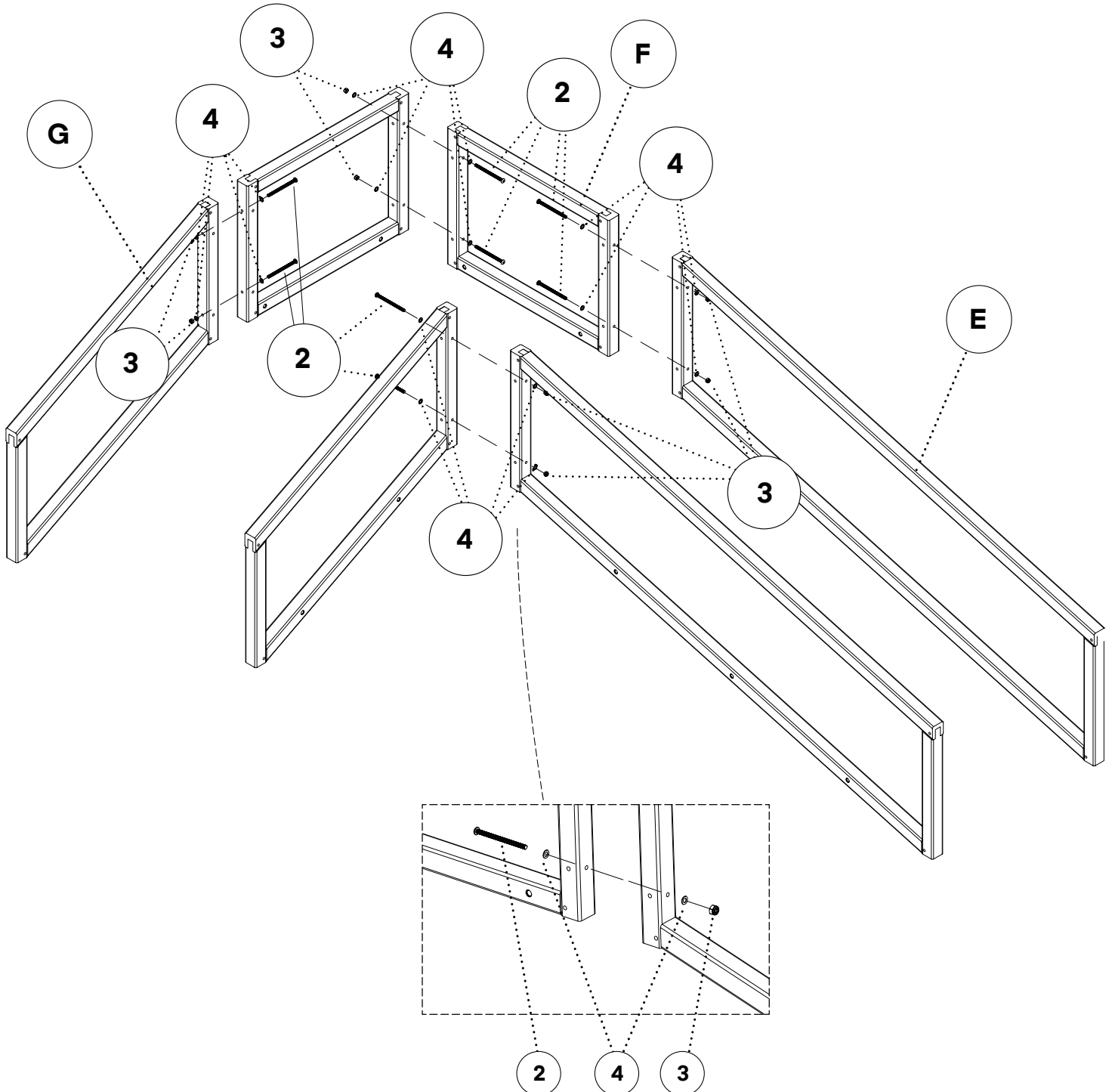

**Step 3:**


**A.** Align the pre-drilled hole at the bottom of each Handrail Metal Rod, pictured below, with the pre-installed bolt on the side of the stairs.


**Step 4:**

**A.** Secure each connection using a M10 Locking Nut (1) and tighten with a M10 Open-end Wrench (8).

**Hardware required (total):**

28 × M10 Locking Nuts (1)

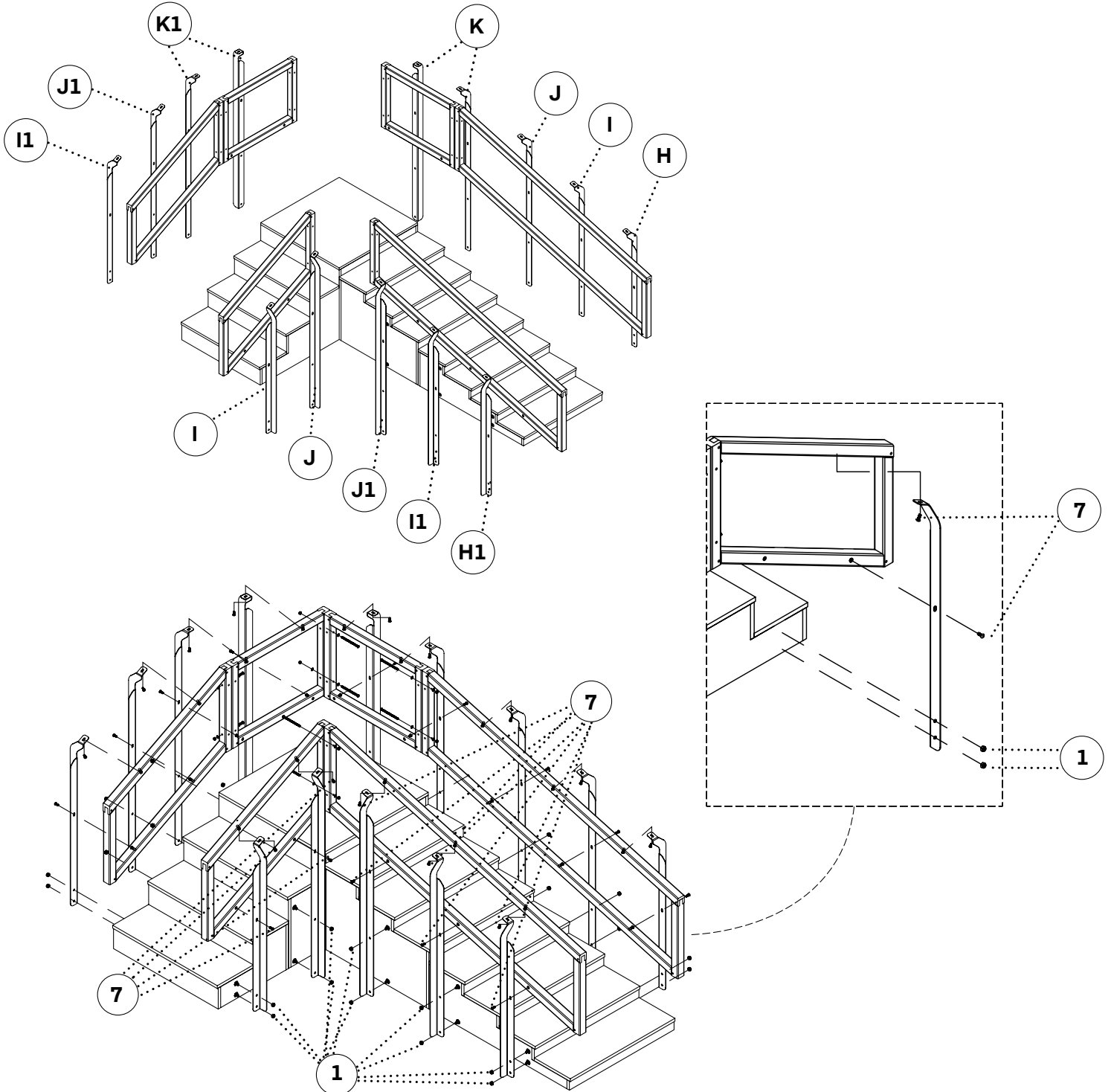

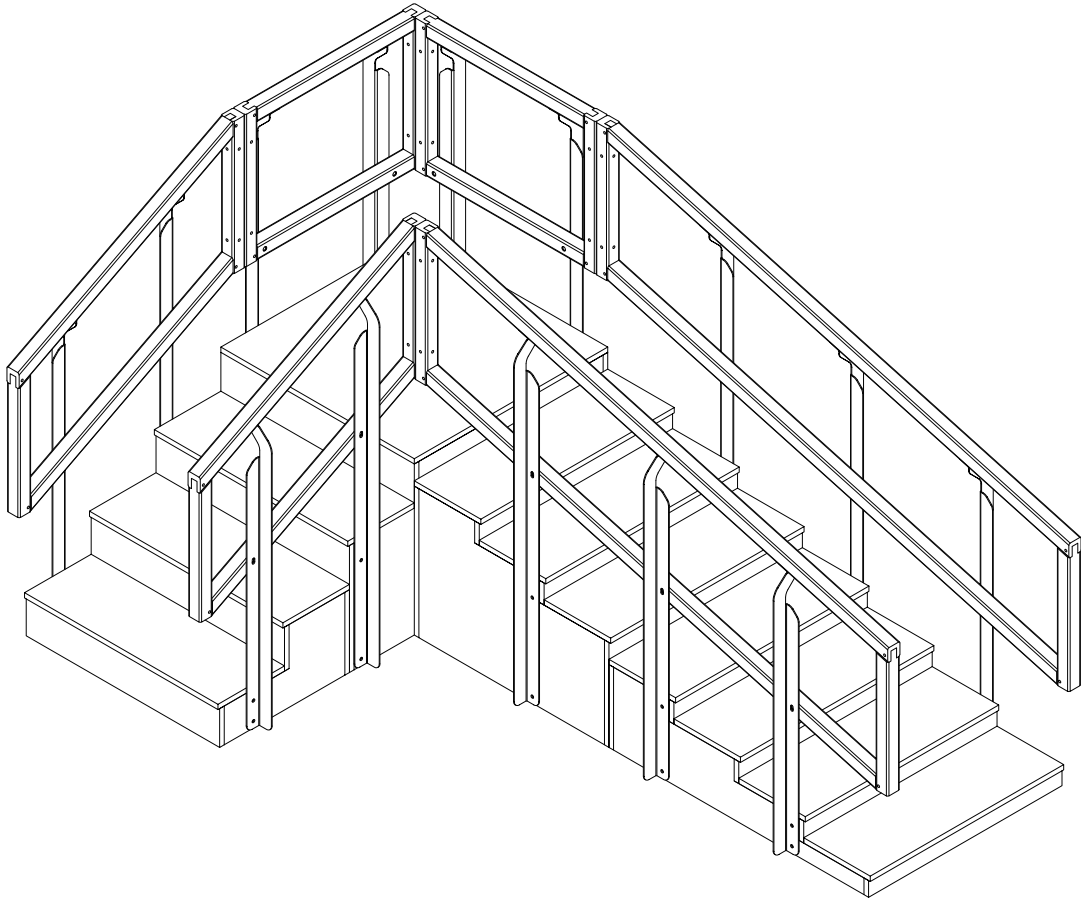





- ⚠ After initially locking all bolts, each bolt and nut must be tightened again to ensure they are secure. Check the guardrail for any movement—if it shakes, continue tightening all bolts and nuts until the guardrail is stable.
- ⚠ If the handrail remains unstable or shakes, discontinue use immediately. Failure to do so may result in injury or damage, for which the user will be solely responsible.

# Safety Warning


A. The user is responsible for preventing damage to this product and injury to patients and/or users.

	WARNING: Indicates potential personal injury.
	Reminder: Keep away from moisture. This device needs to be moisture and waterproof.
	Reminder: Always place the packaging box in the correct direction during transportation, storage, and handling to avoid damage and safety problems caused by incorrect placement.
	Reminder: This product is fragile and needs to be handled with care.

## B. Collapse Risk:

 If the patient's weight exceeds the working load, the stair steps or handrails may break.

## C. Personnel Injured:

 The product is heavy. If you need to transport stairs, please ensure that at least 4 adults are moving it together to avoid accidents.

## D. Usage and Storage Environment Conditions:

a. The ambient temperature requirement for using and storing the equipment is: 0-40°C. If the equipment is exposed to an environment exceeding the specified temperature for a long time, it may cause the risk of plywood deformation or cracking.

b. The humidity requirement for equipment transportation, use and storage environment is: 20%~80%. If the equipment is exposed to an overly dry or humid environment for a long time, it may cause the risk of plywood deformation or cracking, as well as a risk of metal rusting.

c. The metal parts of the equipment are strictly prohibited from contacting liquid water, which may cause metal materials to rust.

## E. Other precautions:

- a. It is strictly prohibited to approach fire sources.
- b. Do not place it in direct sunlight.
- c. Do not place it in high-temperature places such as heaters.

# Care & Maintenance Guide

## **A. Metal and Plywood Maintenance:**

- a.** A dry towel is recommended to wipe metal and plywood.
- b.** If disinfection is necessary, it is acceptable to use hypochlorous acid disinfectant or alcohol with a concentration below 70% to the maximum extent possible. Using highly corrosive disinfectants is prohibited for disinfection. Otherwise, the paint or wood veneer may be damaged.


## **B. Regular Maintenance:**

- a.** Check the steps and handrails every three months to test its stability when pushed horizontally/vertically/laterally. If there is any shaking, tighten the connecting bolts and nuts.

# Warranty

1. Viva Comfort (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 1 year from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.



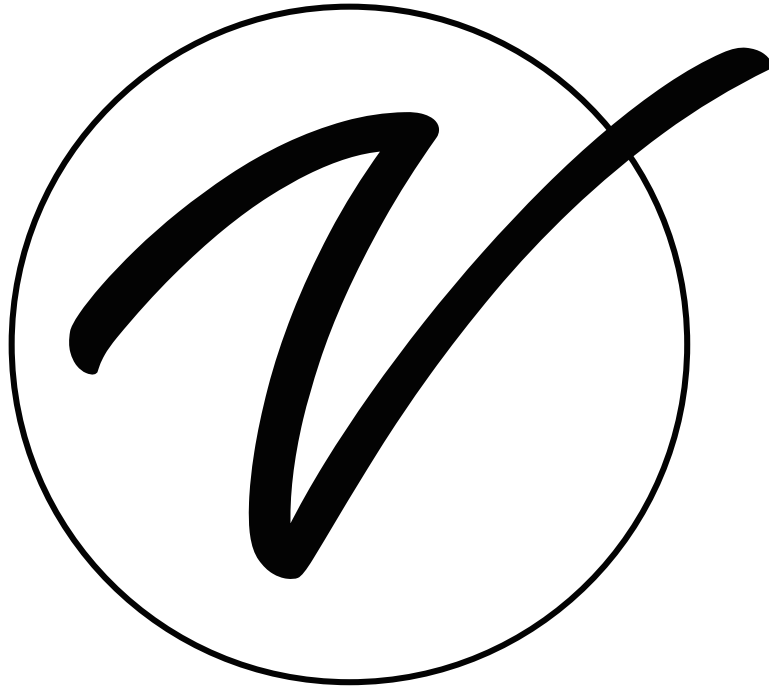

**ADIME906-02-30**

© 2025 Viva Comfort

**vivacomfort.com**